Οδηγίες εφαρμογής - fitting instructions - instrucciones de adaptación - adaptatamento - uiswijzingsinstructies - οδηγίες εφαρμογής - fitting instructions - instrucciones de adaptación - adaptatamento - uiswijzingsinstructies

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ENG

Read carefully these instructions before using the brace. For any doubt, consult always your doctor or the orthopaedic technician.

- Fitting Instructions:**
- The fitting of this orthosis needs the help of an adult.
 - Introduce the arm on the arm support.
 - Pass the fixation cinch strap by the back crossing it by the opposite shoulder.
 - Introduce the shoulder cinch strap through the buckle, adjust it to the necessary height, closing the cinch strap.
 - Provided with a rest lace for the hand.

- Cautions:**
- The doctor is the person qualified to prescribe and decide the treatment duration.
 - The orthopaedic technician is the person qualified to fit the product.
 - For the highest efficiency, you should respect the daily time of use of the product following the doctor indications.
 - This orthosis is designed for the use on one patient. Do not reuse on multiple patients.
 - This orthosis should never be used for any other use different to the one prescribed. Do not allow children to play with this device.
 - You should inform your doctor if you suffer of cutaneous affections. Avoid the use of creams in contact with the orthosis during the treatment.
 - On the pressure points, the skin should not be injured nor irritated.
 - Do not wear the orthosis when bathing, having a shower, etc.
 - Check periodically that the cinch straps are tight and in good conditions (without grease, cuts, rips, etc.).

Side effects:

Possible carotid hypersensitivity syndrome triggered by the use of slings in obese and/or short-necked patients. Avoid excessive pressure of the brace in the carotid sinus region.

Care instructions:

Hand washable, with cold water. Do not bleach. Do not iron. Do not dry clean. Do not dry. Do not tumble dry. Dry on a flat surface. Do not hang out. It is advisable to maintain the velcro free of dirt preventing them from threads, fluffs, etc, in order to keep their adhesion characteristics.

NOTE: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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