Steady SWASH®

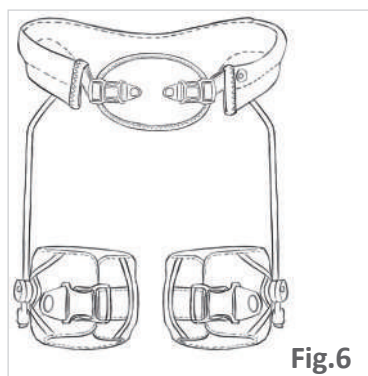
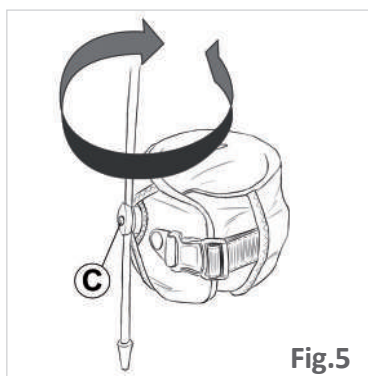
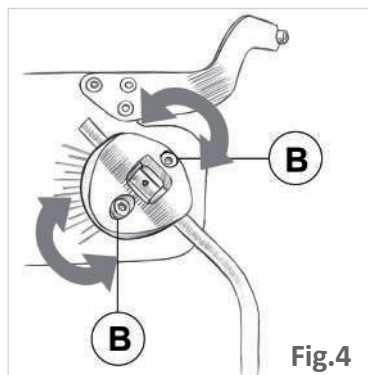
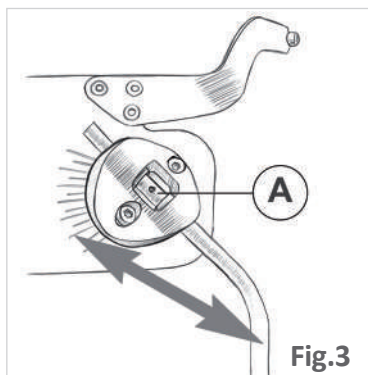
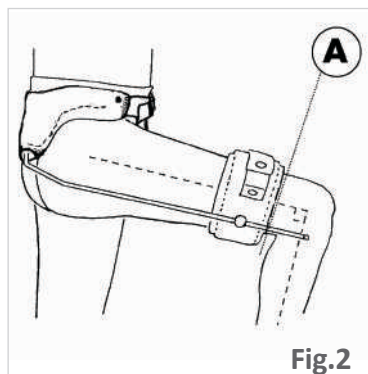
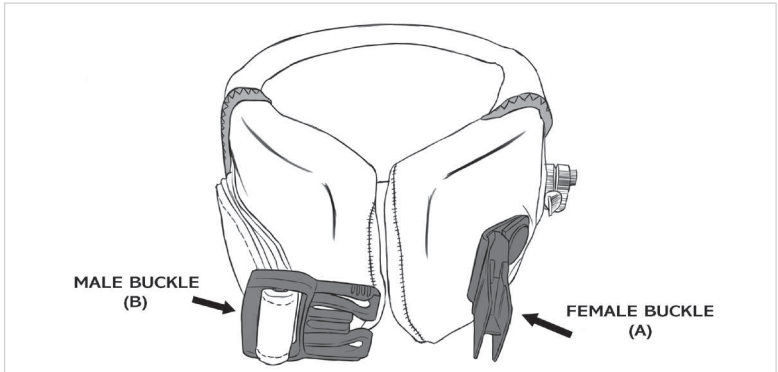
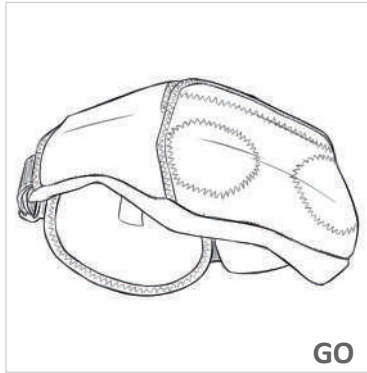
Ego SWASH®

جهاز تقويم التباعد المتغير للورك · Ortesis de abducción variable de cadera · Variable Hüftabduktionsorthese
 · Ortese for variabel hofteabduktion · Variable Hip Abduction Orthosis ·
 Säädettävä lonkan abduktio-ortoosi · Orthèse d'abduction variable de hanche ·
 Varijabilna abdukcijaska ortoza za kuk · Állítható abdukción csípőortézis ·
 Ortesi per abduzione variabile dell'anca · Variabele heupabductie-orthese ·
 Hofteortose med variabel abduksjon · Orteza stawu biodrowego ze zmiennym zakresem odwodzenia ·
 Ortótese de abdução variável de anca · Orteză de abducție reglabilă pentru șold ·
 Тазобедренный контролируемый отводящий ортез · Bedrová ortéza s variabilnou abdukcíou ·
 Höftortos med variabel abduktionsvinkel

تعليمات المستخدم · Instrucciones de adaptación · Anleitung · Tilpasning instruksjon · Fitting instructions ·
 Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások · Istruzioni per
 l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi · Instruções de adaptação ·
 Instruțiuni de reglare · Инструкции по адаптации · Návod na použitie · Utprovningsinstruktion

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Indications:

Cerebral Palsy, Spastic Dipleggia, Spastic Quadriplagia Risk of Hip Dysfunction, Spina Bifida, Night splint, post-botox, postoperative stabilization.

Contraindications:

Dislocated hips. Hip flexion contractures greater than 20° (dynamic or fixed). Abductors too strong for the SWASH™ to handle. Adductor length too short to allow SWASH™ use in the sitting position. Dyskinetic cerebral palsy except as a hip prophylactic orthotic management device. Waist or thigh sizes above those accommodated by the largest SWASH™.

Precautions:

- Read these instructions carefully before fitting the orthosis. If you have any questions, consult your fitter or prescriber before fitting to patient.
- This orthosis must be prescribed by a suitably qualified clinician and correctly adapted to the patient and their needs. Do not make inappropriate modifications to the product.
- This orthosis must not be used for any other purpose than that prescribed. Do not let anyone play with this device.
- Always follow these general instructions of use and any particular guidelines indicated by the prescriber. The doctor should review treatment frequently. If you experience any side effects,

you should immediately inform your doctor.

- You must inform your prescriber if you suffer from any skin conditions. Avoid the use of topical creams during treatment to prevent premature deterioration of the padding. This product is latex free.
- Make sure that the skin in support and pressure areas is not injured or hypersensitive. The pressure of the bands needs to be firm but should not impede circulation or cause discomfort to the user.
- Do not wear the orthosis while bathing, showering or at the beach or swimming pool to avoid damage to the moving parts.
- Before each use, carefully check the condition of all of the components of the orthosis. If any component is not in good condition, it must be replaced with an original part before further use.
- Special attention should be paid to the joints, which must be firmly secured but able to move.
- To avoid the risk of fingers or hands being trapped, do not handle the bars of the orthosis while it is in use. Do not use the orthosis unless all of its parts are in place, including the fabric covers.
- It is essential to maintain proper bodily hygiene and to regularly clean the orthosis. Do not fit the pads or fabric components until they are completely dry.

Assembly of the orthosis:

- **Position the pelvic band** so that the strap fits into the pelvic notch. The side plates can be adapted, but not the lumbar one, as this angle must remain factory set.
- **Fasten the pelvic band front pad:** You should be able to fit only the fingers of one hand between the waist front pad and the child's abdomen. (Fig. 1)
- **Position the thigh cuffs:** Position them on the uprights so when knee is flexed at 90°, they will be approximately 2/3 distal on the thigh. (Fig. 2)
- **Adjust for hip clearance:** Loosen the retaining ring set screw A in the joint assembly and slide each upright in or out until there is just enough distance between them so the uprights do not touch the buttocks and greater trochanters during hip flexion, the bars proximal ends should not touch. (Fig. 3)
- **Adjust abduction angle:** The abduction degree on the hip can be changed loosening the B screws and varying the angle of the joint assemblies (Fig. 4)
- **Check the fit:** Check that the uprights clear the greater trochanters. Repeat step 3 if necessary. Check if abduction is appropriate when sitting, and if ambulatory, when standing and walking. Minor abduction adjustment can be made by rotating the thigh cuffs around the bars. Tighten the C screws. (Fig. 5)
- **Tighten:** Tighten the Allen screws in the joint assemblies and thigh cuffs using the Allen wrenches provided. Reassess the child's function within the orthosis while sitting, and if ambulatory, while standing and walking. (Fig. 6)
- **Finishing:** When fitting is complete, cut the upright excess, the bar end should be at the same level that the bottom of thigh band padding. Ensure the ends are rounded and polished smooth. Then cover the bar ends with the rubber caps. The uprights cannot be adapted, their shape must remain as set at the factory.

Note: There is a professional manual for the practitioners and qualified technicians. The manuals can be downloaded from Allard International web site www.allardint.com, www.allardusa.com, www.allarduk.co.uk.

Disassembly and assembly for washing:

THIGH CUFFS

DISASSEMBLY

1. Unfasten the buckle.
2. Remove the fabric along with the internal padding, starting in the area of the female buckle (A).

Important: during this process, it is not necessary to change the position of the hip joints or remove the bars.

ASSEMBLY

1. Proceed in reverse order by placing the fabric in the area of the female buckle (A).

PELVIC SECTION

DISASSEMBLY

1. Unfasten the belt and open the front closure.

2. Remove the front padding by pulling it.
3. Open the rear window exposing the hip joints.
4. Remove the fabric (the lumbar padding can also be removed for washing).

In the Go model size 6, before removing the fabric, unfasten the hip closures by means of the poppers.

Important: during this process, it is not necessary to change the position of the hip joints or remove the bars.

ASSEMBLY

1. Proceed in reverse order by placing the fabric again. Place the lumbar padding (if it has been removed), ensuring that it is centred.

Product maintenance – washing:

Metal structure: Do not immerse in water. Do not use solvents. Clean the interior surfaces with a damp towel. Do not use grease on the moving parts, as these are self-lubricating.

*Removable fabric parts: Wash at 30°C. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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Luxação da anca, contracturas em flexão da anca >20°, fixa ou dinâmica. Abdutores muito potentes com a ortose SWASH©. Abdutores muito curtos que impessam a sedestação com a ortose SWASH©. Paralisia cerebral discinética, pode ser utilizada como ortose profilática da anca. Tamanho do paciente superior às oferecidas por SWASH©.
PRIM, S.A.