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ENG

Indications:

NP139 NEOPRAIR Closed knee support: Joint pain, arthritis, osteoarthritis, post injury.
NP133 NEOPRAIR Knee support with patella pad / NP137 NEOPRAIR Knee support with patella pad and splints / NP140 – NP141 NEOPRAIR Knee support with polycentric joint – Wraparound knee support with polycentric joint: Joint pain, bursitis, tendinitis, arthritis, osteoarthritis, patellar injuries, lateral knee ligament injuries, Osgood-Schlatter disease, chondromalacia patella, patellar subluxation, post injury, post surgery.

NP136 NEOPRAIR Knee support with splints and straps: Joint pain, bursitis, tendinitis, arthritis, osteoarthritis, patellar injuries, lateral knee ligament injuries, mild capsule/ligament instability, post injury, post surgery.
NP115 NEOPRAIR Cross-strap ankle support / NP114 NEOPRAIR Figure-of-eight ankle support with stabilisers: Minor sprains and prevention of sprains while playing sport, tendinitis.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Treatment with the orthosis should be regularly monitored by your doctor.
- If any side effects are experienced, your doctor should immediately be informed.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components (straps, Velcros, silicone pads, etc.) should be regularly checked to ensure that they are in good condition.
- No let children play with this device.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath, shower, etc.

Instructions for proper fitting:

During periods of use, follow the advice of the professional who recommended and/or prescribed the orthosis. To obtain the maximum benefit from the orthosis under different conditions and to prolong the life of the product, it is essential to select the right size for each patient or user. As excessive compression can lead to intolerance, we recommend adjusting the compression to the optimum degree. Fit the product to the joint in question, making sure that it is properly aligned to enable correct movement of the muscles. Adjust the product to ensure a snug

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