neoprair

Protecciones de neopreno transpirable
Breathable neoprene supports
Attelles de Support en neoprene aéré
Protezioni in neoprene traspirante
Proteções de neoprene respirável

NPOS703 · NPOS136
NPOS120 · NPOS147 · NPOS132 · NPOS111

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení
Anleitung · Tilpasning instruktion · Οδηγίες εφαρμογής · Fitting instructions · Sovitusohjeet
Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások
Istruzioni per l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi
Instruções de adaptação · Instruțiuni de reglare · Инструкции по адаптации
Návod na použitie · Utprovninginstruktion

 PRIM

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ENG

Description and intended use:

NPOS136 NEOPRAIR Knee support: Wraparound knee orthosis with 4 stabilising stays, open in the popliteal and patellar area. Made of three-layer breathable, elastic and padded material.

NPOS703 NEOPRAIR Short knee support: Compact wraparound knee orthosis with 4 stabilising stays, open in popliteal and patellar area. Made of three-layer breathable, elastic and padded material.

NPOS111 NEOPRAIR Ankle support: Wraparound ankle orthosis with figure-of-8 closure system. Made of three-layer breathable, elastic and padded material.

NPOS147 NEOPRAIR Thigh support / NPOS120 NEOPRAIR Calf support: Thigh orthosis for quadriceps and calf orthosis for the calves. Made of three-layer breathable, elastic material.

NPOS132 NEOPRAIR Patellar strap: Infrapatellar support orthosis with removable silicone pad. Made of three-layer breathable, elastic material.

Indications:

NPOS136 NEOPRAIR Knee support: Joint pain, bursitis, tendinitis, arthritis, osteoarthritis, patellar injuries, lateral knee ligament injuries, Osgood-Schlatter disease, chondromalacia patella, patellar subluxation, post injury, post surgery.

NPOS703 NEOPRAIR Short knee support: Joint pain, tendonitis, minor knee instability and patellofemoral instability. Stabilisation during sporting activities or situations involving exertion.

NPOS111 NEOPRAIR Ankle support: Minor sprains and prevention of sprains while playing sport, tendinitis.

NPOS147 NEOPRAIR Thigh support / NPOS120 NEOPRAIR Calf support: Muscle tears (quadriceps thigh support) and (calf support), muscle strains, post injury.

NPOS132 NEOPRAIR Patellar strap: Osgood-Schlatter disease, chondromalacia patella, patellar tendon tendinitis, sporting activities involving forced extension of the knee.

Contraindications:

- Do not use on open wounds with swelling.
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the supplier you purchased it from before use.
- Always follow these general instructions for use and any particular guidelines indicated by your doctor.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor should consider the use of topical creams in conjunction with the orthosis.
- The skin should not be damaged or hypersensitive in the areas where pressure is applied.

- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting and adjustment of these accessories must be carried out by qualified personnel (doctor, orthopaedic technician etc.).
- For the orthosis to function correctly, its components must be regularly checked to ensure they are in good condition. The medical personnel responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- Care should be taken to clean the Velcro quick-release fasteners to maintain their functionality, removing any materials that may have adhered to them.
- Do not use in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain adequate personal hygiene.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.
- Do not let children play with this product.
- Any serious incidents involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient resides.

Fitting instructions:

When using, follow the advice of the professional who recommended and/or dispensed the product. To gain the highest degree of therapeutic effectiveness for the different conditions and to prolong the life of the product, it is essential to select the right size for each patient or user. As excessive compression can lead to intolerance, we recommend adjusting the compression to the optimum level. Fit the product to the joint in question, making sure that it is properly aligned to enable correct movement of the muscles. Adjust the product to achieve a snug fit with no discomfort.

NPOS136 NEOPRAIR Knee support / NPOS703 NEOPRAIR Short knee support: Fit the orthosis by centring the opening in the knee support over the kneecap, ensuring that the Velcro fastener bearing the logo is positioned in the thigh area.

NPOS111 NEOPRAIR Ankle support: Fit the orthosis by first fitting the grey area to the ankle using the eyelet and Velcro straps. Next, adjust the straps over the malleoli, exerting the desired pressure.

NPOS147 NEOPRAIR Thigh support / NPOS120 NEOPRAIR Calf support: Wrap around the thigh or calf and adjust the compression with the Velcro fasteners.

NPOS132 NEOPRAIR Patellar strap: Centre the silicone pad on the inside of the orthosis and position just below the kneecap and above the tuberosity of the tibia. Tighten the straps to close, adjusting the pressure with Velcro fasteners.

Product maintenance - washing instructions:

Before washing the support, fasten all of the Velcros and detach the removable components (silicone pads, splints, palmar stays, etc.). Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose the orthosis to direct heat sources such as stoves, heaters, radiators and direct sunlight.

Note: Any serious incidents involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.