MPK300 • MPK400 • MPK401

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·
Tilpasning instruktion · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions ·
Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·
Istruzioni per l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi ·
Instruções de adaptação · Instruțiuni de reglare · Инструкции по адаптации ·
Návod na použitie · Utprovningsinstruktion · Navodila za prilagoditev · Uždejimo instrukcijos ·
Pielāgošanas instrukcijas

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ENG

Description and intended use:

MPK300: Clavicle immobiliser made from breathable honeycomb fabric on the outside and absorbent terrycloth with a pleasant feel on the inside. The back is composed of a dorsal pad that provides support and comfort thanks to its interior piece of foam padding.

The straps are made from rigid velour to keep the shoulders back and achieve the correct immobilisation, and are also covered with a foam protection to ensure greater comfort and prevent rubbing.

MPK400: Sling made from breathable honeycomb fabric on the outside and absorbent terrycloth with a pleasant feel on the inside. The design has an ergonomic shape for the elbow and forearm, plus cushioned padding for the neck and shoulder to prevent discomfort.

It features an immobilising attachment that wraps around the arm and trunk allowing the shoulder joint to be immobilised.

Indications:

MPK300: Treatment for stable fractures of the clavicle, correctly aligning the fractured sections. Other disorders where it is necessary to keep the shoulders back. Posture reminder.

MPK400: Post-operative or post-injury immobilisation treatment. Injury treatment that requires immobilisation of the arm or shoulder (if used with the attachment). Soft tissue injuries such as burns and infections.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your

doctor or the shop where you purchased it.

- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

Side effects:

Possible carotid hypersensitivity syndrome triggered by the use of slings in obese and/or short-necked patients. Avoid excessive pressure of the brace in the carotid sinus region.

Instructions for proper fitting:

MPK300: Once the appropriate size has been chosen for the patient, place the dorsal pad on the back at the height of the shoulder blades, pass the straps under the armpits and backwards over the shoulders and fasten first on one side (Fig. 1) and then on the other (Fig. 2). The lengths of the straps then need to be adjusted through the D-rings located at the bottom of the dorsal pad. Tighten the straps by pulling evenly until the shoulders are pushed back and correctly immobilised (Fig. 3).

The straps can be trimmed if they are too long by following these steps: Unfasten the lower part of the straps (Fig. 1), detach the velcros (Fig. 2) and then trim the straps to the necessary length (Fig. 3). Lastly, replace the velcros (Fig. 4) and fasten the straps (Fig. 5).

MPK400: Once the appropriate size for the patient has been chosen, place the forearm into the sling and then pass the strap around the back of the neck and shoulder opposite the arm to be treated. The pad should be protecting the area of the neck and shoulder. Lastly, adjust the length of the strap and fastening of the thumb with the velcro system.

MPK401 OS attachment: This product comes in one size. For fitting, once the MPK400 sling is on the patient, lift the velcro (Fig. 1), attach the end of the attachment to the grey velour area (Fig. 2) and then fasten the velcro to the attachment's own fastening (Fig. 3). Finally, wrap the attachment around the arm that needs to be immobilised so that the shoulder is also immobilised.

Product maintenance - washing instructions:

Before washing, close all of the velcros. Hand wash with warm water (max. 30°C). Use neutral detergent. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

For drying, use a towel to absorb as much moisture as possible and then leave to dry at room temperature. If the orthosis is not well rinsed out, detergent residue can remain and cause irritation to the skin and damage to the product. Rinse the orthosis with plenty of water to remove any remaining detergent which could damage the product or cause skin irritation.

Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc. Do not use alcohol, creams or solvents for cleaning.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.