ORTHO Prim

Corrector Nocturno Juanetes

NIGHT-TIME
BUNION CORRECTOR

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Ref.P435

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ENG - P435 - NIGHT-TIME BUNION CORRECTOR: *Night alignment orthosis.***INDICATIONS:**

Treatment of Hallux Valgus (bunions).

INSTRUCTIONS FOR USE:

- Before fitting the orthosis, check the markings on the plastic part: it will display either an "R" for the right side or an "L" for the left side.
- Position the padded head of the orthosis between the big toe and the second toe, ensuring that the pad makes contact with the big toe. The two extensions of the orthosis should rest on the top (instep) and the bottom (sole) of the foot (refer to Fig. 1)
- Adjust the bunion strap to apply the necessary pressure for a secure fit (see Fig. 2).
- Next, adjust the strap in the instep area until the product fits correctly on your foot (see Fig. 3).
- If needed, readjust the strap around the bunion area (see Fig. 2).
- When correctly positioned, the product should resemble (Fig. 4).

RECOMMENDATIONS:

In areas of support where pressure is applied, the skin should not be damaged or hypersensitive. The use of these products with topical medicinal products should be supervised by the appropriate medical personnel.

WARNINGS:

- Follow the usage and fitting instructions that come with the orthosis.
- Follow the washing and drying guidelines specified in the instructions or labelling.
- Do not walk or stand with the corrector in place. Pain should not interfere with rest.
- Consult a specialist when necessary.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, cover or protect the orthosis from the water.
- Before fitting the product, ensure that it is positioned on the correct side.
- After washing, apply talcum powder to the area that comes into contact with your socks or stockings to reduce friction. Do not apply talcum powder to the area in contact with the skin.

CONTRAINDICATIONS:

- Do not use for prolonged periods without medical supervision.
- Do not use the products on open wounds, cuts, blisters or any type of irritation on the feet.
- Do not use adhesive products for prolonged periods without allowing the skin to breathe.
- If you experience worsening symptoms or new problems after starting to use any of these products, discontinue use and consult a doctor or podiatrist.
- If you suffer from loss of sensation in your feet due to conditions such as diabetic neuropathy, use the products with caution and under the supervision of a doctor or podiatrist to avoid undue pressure or discomfort.
- If you have any doubts about the suitability of the product, consult a doctor or podiatrist.

WASHING AND STORAGE INSTRUCTIONS:

Before washing, remove the straps and close all the Velcros. Clean the plastic components with a damp cloth and wash the fabric sections by hand with mild soap, then dry thoroughly. Do not use bleach. Do not dry clean. Do not iron. Do not tumble dry. We recommend storing the product in its original packaging.

NOTE: Any serious incidents involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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