ATXOS700

ATXOS703

airtex

Rodillera y rodillera corta

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Description and intended use:

Knee braces with lateral straps for knee stabilisation. Elastic and breathable knee braces with adjustable compression. Designed to fit any knee contour. Fits left or right.

Indications:

ATXOS700 - Knee brace indicated for the treatment of:

Minor instability of collateral ligaments, patellofemoral instability, congenital or acquired instabilities. Joint pain and inflammation associated with osteoarthritis, arthritis and tendonitis. Post-surgical knee treatment, rehabilitation processes.

ATXOS703 - Short knee brace indicated for:

Joint pain, tendonitis, minor knee instability and patellofemoral instability. Stabilisation during sporting activities or situations involving exertion.

Contraindications:

- Do not use on open wounds with swelling.
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop that sold you the product.
- For the orthosis to serve its purpose, it must be correctly adjusted to the patient's morphology. The first fitting and adjustment must be carried out by qualified personnel (doctor, orthopaedic technician, etc.).
- Always follow these general instructions of use and any particular guidelines indicated by your general practitioner.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- To ensure they function correctly, clean the hook-and-loop fastenings regularly, removing any material that may have stuck to them.
- Some of the materials that make up the product are not flame retardant. Therefore, its use is not recommended in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain proper hygiene.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.
- Do not let children play with this device.

Instructions for proper fitting:

Before using for the first time, remove the transparent film from the non-slip area (ATXOS700)

- Centre the opening of the knee brace over the patella Fig. 1.
- Adjust the lower strap over the calf Fig. 2 and the upper strap over the thigh Fig. 3.
- For maximum efficacy, compression should be adjusted to a firm but comfortable level.

Product maintenance - washing instructions:

To wash the fabric parts, fasten the micro-hooks and wash by hand with a mild soap (max. 30°C). Do not use bleach. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.