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ENG

Carefully read these instructions before using the orthosis.

Indications:

Clubfoot, tibial rotation, knee valgus.

Fitting instructions.

1. Positioning the cuffs.

Slightly loosen the screws (61) and (81). Put the orthosis on and adjust the position of the cuffs (7-9) and the heel assembly (4) according to the morphology of the baby's lower limb. This adjustment must be done with the knee flexed 80°-90°. When the correct position is obtained, tighten the screws (61) and (81). The flexion axis of the knee part (8) must be aligned with the child's knee flexion axis. If you use a moveable knee joint, passively flex it all the way and check that the cuffs do not pinch the fleshy parts near the popliteal fossa. If you use an adjustable knee joint, passively flex it using the Allen key to turn the screw (82) clockwise. If there is pinching, readjust both cuffs. Do not forget to adjust the cuffs at each visit according to the growth of the child.

2. Fixing the foot on the sole.

After having completely loosened the screw (42), remove the sole assembly. Fix the foot on the sole using non-elastic adhesive bands. The foot will be fixed according to the methods that are normally practised, taking care that the heel and the forefoot are perfectly in contact with the sole. Never cover the screws (31-32) of the multidirectional hinge. Replace the sole on the heel assembly (43) using the screw (42) and adjust to the desired position according to the disorder to be treated and its various deformity components. When the correction position is obtained, tighten all the screws.

3. Possible corrections.

Varus and the valgus of the hindfoot

Loosen the screws (51-52) of the multidirectional hinge (5) and incline the heel assembly (4) towards the left or the right. When the desired corrected position is obtained, tighten the screws.

Abduction and adduction of the foot

Loosen the screw (41) and adjust the position of the foot to align the global rotation axis of the foot with the vertical axis of the tibial diaphysis by moving the part (43) forward or backward and then tighten the screw (41). Loosen the screw (42) without removing it, slightly raise the sole assembly (1-2-3-44) and turn it towards the right or the left. When the desired corrected position is obtained, pull the sole assembly back in its housing and tighten the screw (42).

Equinus

Loosen the screw (61) and the screws (51-52) of the multidirectional hinge (5), pull the assembly (6-5-4) vertically downwards in order to stretch the Achilles tendon. Simultaneously dorsiflex the foot. When the desired corrected position is obtained, tighten the screws (61) and (51-52) of the multidirectional hinge (5).

Abduction and adduction of the forefoot

Loosen the screws (31-32) of the part (3) and abduct or adduct the forefoot in relation to the hindfoot. When the desired corrected position is obtained, tighten the screws of the multidirectional hinge (3).

Pronation and supination of the forefoot

Loosen the screws (31-32) of the part (3) and pronate or supinate the forefoot in relation to the hindfoot. When the desired corrected position is obtained, tighten the screws of the multidirectional hinge (3).

ENG

4. Positioning a Bebax

Completely loosen the screw (61) and withdraw the foot assembly from its housing. Completely loosen the screws (51-52) of the multidirectional hinge (5). Take off the sole assembly by completely loosening the screw (42). Completely loosen both screws (31) in order to release the metal part (44) from the sole. Replace the short Achilles part (6) with the long Achilles part (60) provided separately with the orthosis. Replace the short multidirectional hinge (5) with the long multidirectional hinge (50) provided separately with the orthosis. Fix the part (44) on the back part of the multidirectional hinge situated under the Bebax orthosis using the screws of the orthosis (31). Put the foot assembly back in its housing and tighten the screw (61). The orthosis is ready for the corrections.

Precautions

Always follow these general instructions and the special indications prescribed by your doctor. A qualified doctor is the only person who can prescribe and determine the duration of the treatment. For maximum effectiveness, follow the instructions specified by your doctor regarding the daily duration of use. For complete therapeutic results, you must use the correct size of the product, and the first fitting should be done by a qualified professional. Once fitted, do not try to modify the adjustment of the orthosis. Your doctor will increase or reduce the correction of the orthosis during consultations. He or she is the only person who can determine when the treatment is finished. For any doubts, always consult your doctor or the orthopaedic technician who fitted the orthosis. Inform your doctor if you notice any secondary effect. This orthosis should never be used for any use that is different than the one prescribed. Do not allow children to play with this orthosis. This orthosis must not be used by another patient. This orthosis can be used with Bebax orthosis (optional, not included/consult sizing). The condition of the orthosis must be controlled periodically by a responsible adult. Do not use creams during the treatment in order to avoid its premature deterioration. The child must not wear the orthosis when bathing, showering, etc.

Maintenance instructions and lifespan

Wash with a cloth and lukewarm water. Do not use bleach. Wipe dry with a cloth. Under normal usage conditions, the lifespan of this orthosis is 6 months.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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