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ENG

**Indications:**

To immobilize the shoulder, as symptomatic treatment of shoulder contusions or before surgery (Mumford surgery, collarbone fracture or surgery, shoulder surgery and fracture of proximal third of humerus).

**Counter indications:**

Unstable fractures, or fractures which present neurovascular complications, or which need surgery to achieve stabilization.

**Precautions:**

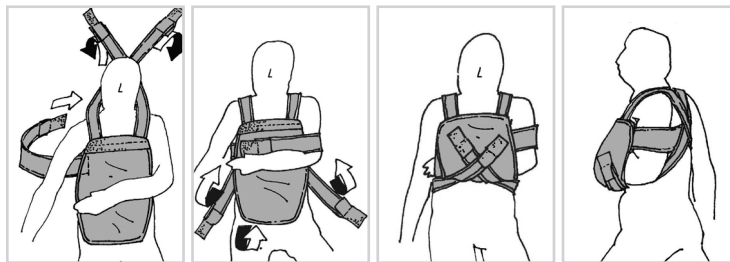
- Read carefully these instructions **before** using this orthosis. If you have any doubt, consult always your doctor or the establishment where you have acquired it before beginning to use it. For a complete therapeutic effect, the patient should wear the orthosis of the right size. The first time it is used, it should be fitted by qualified personnel. Your doctor is the only person qualified to prescribe this orthosis and to decide the treatment duration. For the highest performance, the patient should respect the daily use and treatment duration, following the prescriptions of the practitioner.
- This orthosis must never be used for any other different use for which it has been prescribed. Don't allow the children to play with this device. This orthosis should not be used by more than a single patient.
- You should inform your doctor about if you suffer skin problems. Avoid using creams, lotions or powders during the treatment. In the areas under pressure, the skin should not be injured nor be hypersensitive.
- For hygiene and comfort reasons, it is advisable to use a cotton garment, preferably without seams, under this orthosis. It is also advisable a periodic washing of this orthosis.
- Do not use the brace during the bath or a shower, etc.
- Check periodically that the straps are tight and in a good working condition (without grease, cuts, etc.).
- Special care must be paid of to the circulating state of the upper extremities, and loosen the orthosis if a vascular compression problem is suspected.

**Side effects:**

Possible carotid hypersensitivity syndrome triggered by the use of slings in obese and/or short-necked patients. Avoid excessive pressure of the brace in the carotid sinus region.

**Fitting instructions:**

1. It is advisable for the patient to rely on the help of another person for the placement of this orthosis.
2. Put the orthosis on, the fracture side first and then the other one, adjust the orthosis until the shoulders be straight, but not hyper-extended in excess.
3. Adjustments can be done fastening or loosening the straps in the back buckles. If necessary, cut the straps.
4. GUMIFIX allow the active and passive of the elbow mobility by open without remove the front part of the orthosis, allowing the mobility of forearm and no allowing the shoulder activity. The drawings teach how to place the orthosis and how by open the front part of the orthosis the elbow is able to get passive and active mobility without shoulder activity.

**Product maintenance - washing instructions:**

Hand wash in warm water. Do not bleach, Do not iron. Do not dry wash. Do not tumble dry. Dry on an horizontal surface. Do not tend.

**Sizes:**

One size. Right and Left.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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