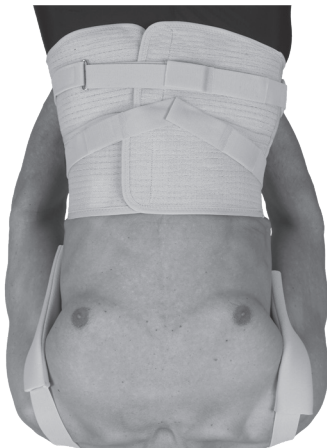
KNIGHT DUO

TAYLOR DUO

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ENG

Read carefully these instructions before using the orthosis. For any doubt, consult always the doctor or the establishment where the orthosis has been acquired.

Fitting instructions:

It is advisable to put on the orthosis in front of a mirror. For hygiene and comfort reasons, it is advisable to wear a cotton garment, preferably with no seams, underneath the orthosis.

Taylor Duo orthosis:

- The orthopaedic technician should adapt the strip to the patient.
- Place the lower frame at the trochanters height, centring the back (Fig. 1).
- Close the lower frontal band (Fig. 2).
- Close the upper frontal band, adjusting the lacing, if it were necessary (Fig. 3).
- Thread the lower cinch strap, cutting, if it were necessary. Place the closing Velcro® on the free end. Repeat the same operation with the upper cinch strap (Fig. 4).
- Cross back the straps, passing them through the slide and adjust their length. The help of a second person is convenient for this operation (Fig. 5).
- Verify it is centred and correctly adjusted in height (Fig. 6).
- Cut the remaining cinch strap, place the floating Velcro® on the new end (Fig. 7).

Knight Duo orthosis:

- The orthopaedic technician should adapt the strip to the patient.
- Place the lower frame at the trochanters height, centring the back (Fig. A).
- Close the lower frontal band (Fig. B).
- Close the upper frontal band, adjusting the lacing, if it were necessary (Fig. C).
- Thread the lower cinch strap, cutting, if it were necessary. Place the closing Velcro® on the free end. Repeat the same operation for the upper cinch strap (Fig. D).
- Check it is centred and correctly adjusted in height (Fig. E).
- Cut the remaining cinch strap, place the floating Velcro® on the new end (Fig. F).

Cautions:

- The doctor is the person qualified to prescribe the orthosis and decide the duration of the treatment.
- The orthopaedic technician is the qualified person to shape the strip accordingly with the contour required and to fit of the orthosis.
- For a correct effectiveness, you should respect the daily time of use following the doctor or your orthopaedic technician indications.
- For a correct function, you should use the correct size of the orthosis.
- Orthosis of personal and untransferable use. It should not be used for the use of another patient.
- This orthosis should never be used for any other use different to the one prescribed. Do not allow children to play with this device.
- You should inform your doctor if you suffer of some skin affections. Avoid the use of creams, in contact with the orthosis, during the treatment.
- On the pressure points, the skin should not be injured nor irritated.
- For hygiene and comfort reasons, it is advisable to wear a cotton garment, preferably with no seams, underneath the orthosis.
- Do not wear the orthosis when bathing or having a shower, etc.
- Check periodically that the cinch straps are tight and in good conditions (without grease, cuts, rips, etc.).

Maintenance instructions:

Taylor: Take off the back Velcro®, slide the cinch straps by the frame until removing it completely.

Knight: Disassemble the screws of the upper band with the Allen key, which is provided with the product. Slide the fabric by the frame, until removing it completely.

Both models: Close all the Velcro® on the surface of the garment before washing it. Hand washable with cold water. Do not bleach. Do not iron. Do not dry-clean. Do not tumble dry. Dry on a flat surface. Do not hang out.

It is advisable to take care of the Velcro® cleaning in order to maintain their adhesion characteristics, eliminating threads, fluffs, etc.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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