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ENG

Instructions of Use for Elcross 28181P Ankle brace:

Read carefully these instructions before using the orthosis. For any doubt, consult always the doctor or technical orthopaedist.

Indications:

Pull-on style with elastic band in 8 to allow flexo-extension, avoiding pronation and supination. Prevention of ankle sprains. Treatment of chronic or acute ankle sprains. Slight ankle injuries.

Precautions:

- Do not tighten too much the ankle brace. Remove it immediately if you observe any change on the skin colour, or when swarming sensation or if you observe any other problem of blood circulation or skin irritation.
- The doctor is the only person who can prescribe and decide the duration of the treatment.
- The fitting of this brace should be done by your technical orthopaedist. For a correct effectiveness, you should respect the daily time of use following the doctor or your orthopaedic technician indications.
- For a correct function, you should use the correct size of the orthosis.
- Orthosis of personal and untransferable use. It should not be used for the use of another patient.
- This orthosis should never be used for any other use different to the one prescribed. Do not allow children to play with this device.
- You should inform your doctor if you suffer of some skin affections. Avoid the use of creams, in contact with the orthosis, during the treatment.
- On the pressure points, the skin should not be injured nor irritated.
- Do not wear the orthosis when bathing or having a shower, etc.
- Check periodically that the cinch straps are tight and in good conditions (without grease, cuts, rips, etc.)

Fitting instructions:

- Introduce the foot by the ankle brace like a sock.
- Heel should rest in the opening.
- To close the ankle brace, take the cinch strap and make an 8 all around the foot until the required support.

Sizes:

Right and left (R, L). Measurement: ankle perimeter.
S (20-25cm)
L (25-30cm)

Product maintenance - washing instructions:

Machine washable at 40°. Do not bleach. Do not iron. Do not dry clean. Do not tumble dry. Close all the velcro straps before washing it to avoid non-covered closures to damage other washed materials.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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