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ENG**INDICATIONS:**

Arthritic and post-surgical support.

CAUTIONS FOR THE PATIENT:

- Read carefully these instructions before fitting this orthosis.
- The doctor is the person qualified to prescribe the orthosis and decide the duration of the treatment. A healthcare professional is the qualified person to fit the product.
- For a correct effectiveness, you should respect the daily time of use following the doctor or your orthopaedic technician indications. For a correct function, you should use the correct size of the orthosis.
- This orthosis is designed for use on one patient. Do not reuse on multiple patients.
- This orthosis should never be used for any other use different to the one prescribed. Do not allow children to play with this device.
- You should inform your doctor if you suffer of some skin affections. Avoid the use of creams, in contact with the orthosis, during the treatment.
- On the pressure points, the skin should not be injured nor irritated. Do not wear the orthosis when bathing or having a shower, etc.

FITTING INSTRUCTIONS:

Introduce the thumb through the hole. Adjust and close the Velcro straps on the wrist. Place the first interdigit between the fore and middle fingers positioning them on the hook Velcro support of the orthosis. Place the second interdigit between the middle and third fingers positing them on the hook Velcro of the first interdigit. Proceed the same with the third and fourth interdigit.

SIZES:

Measurement: arm palm circumference. Side Right and Left available.

NOTE: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

S	16 - 18 cm
M	18 - 21 cm
L	21 - 25 cm

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PRIM, S.A