ELCROSS light

Fajas Elcross Light	2131
Elcross Light Rückenstützbandagen	2132
Elcross Light Korsetter	2133
Elcross Light back supports	2134
Elcross Light Tukiliivit	2137
Ceintures de Soutien Elcross Light	2132N
Corsetti Elcross Light	2133N
Elcross Light Rugbandagen	2137N
Elcross Light Ryggbelter	
Sznurówki Elcross Light	
Cintas Elcross Light	
Elcross Light Gürdel	

Instrucciones de adaptación · Anleitung · Tilpassning instruksjon · Fitting instructions ·
Sovitusohjeet · Instructions d'adaptation · Istruzioni per l'adattamento ·
Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi ·
Instruções de adaptação Utprovningsinstruksjon

ES

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ENG

Indications:

For all process in which a spinal immobilisation is required.

Precautions:

- Read carefully these instructions before using the brace. For any doubt, consult always your doctor or the establishment where you have acquired this brace.
- A qualified medical practitioner is the only person who can prescribe and decide the duration of the treatment.
- The technical orthopaedist is the only person qualified to shape the stays accordingly with the required circumference and to fit the back support
- For a higher efficiency, follow the instructions specified by your medical practitioner regarding the daily time of use.
- For complete therapeutic results, you must use the correct size of the back support.
- This brace should never be used for any other use different to the one prescribed. Do not allow children to play with this brace.
- If you notice any side effects, you must contact immediately your doctor. You should inform your doctor if you suffer of skin affection. For maintaining the life of the product, avoid the use of creams during the treatment.
- On the pressure points, the skin should not be injured nor irritated.
- Do not wear the brace when bathing or having a shower, etc.
- Check periodically the condition of the straps to ensure that they are free from grease, damage, etc.

Instructions for proper fitting:

It is recommended to put the back support on in front of a mirror. For hygienic and comfort reasons, it is advisable to wear a cotton garment underneath your back support.

Elcross Light back supports have been developed with a breathable material. They are provided with frontal or lateral (mod. 2134) internal velcro closures (A) designed ergonomically to help putting on and off the back support.

They also have lateral velcro (C) where the lateral straps (B) could be provisionally placed before fixing them on the frontal closure of the velcro loop (A) of the back support.

Elcross Light sacrolumbar back support models 2131, 2132, 2132N, 2133, 2133N (Fig. 1):

- Make sure that the lateral straps (B) are placed on the lateral velcro (C).
- Take the frontal velcro (A) off the back support. Place the Elcross Light back support to the adequate height and position, without making tension.
- Fit the back support correctly and close the frontal velcro (A). Take the lateral straps (B) off the lateral velcro (C) and pull them with the same tension, from the rear to the front. Close the lateral straps (B) over the frontal closure of velcro loop (A).

Elcross Light back support for pendulous abdomen model 2134 (Fig. 2):

- Make sure that the lateral straps (B) are placed on the lateral velcro (C).
- Take the lateral velcro (A) off the back support. Place the Elcross Light back support to the adequate height and position, without making tension. In this case, considering that this is a back support for pendulous abdomen, the back support has to be placed with the velcro closure laterally (A), as the front part is indicated to lodge the abdomen.
- Fit the back support correctly and close the lateral velcro (A).
- Take the lateral straps (B) off the lateral velcro (C) and pull them with the same tension, from the rear to the front. Close the lateral straps (B) over the frontal closure of velcro loop (A), make sure that they stay under the abdomen.

Elcross Light dorsolumbar back support model 2137, 2137N (Fig. 3):

- Make sure that the lateral straps (B) are placed on the lateral velcro (C).
- Adjust the posterior straps (E) of the back over the shoulder and then pass them under the axillas. Pull backwards and cross them by the back. Adjust them over the velcro loop of the lateral straps (B) which must be placed over the lateral velcro (C) of the back support.
- Take the frontal velcro (A) off the back support. Place the Elcross Light back support to the adequate height and position, without making tension.
- Fit the back support correctly and close the frontal velcro (A).
- Take the lateral straps (B) off the lateral velcro (C) and pull them with the same tension, from the rear to the front. Close the lateral straps (B) over the frontal closure of velcro loop (A).
- Take the rear straps off the back (E) from the lateral straps (B) and pull them with the same tension, from the rear to the front. Close the rear straps of the back (E) over the the frontal closure of velcro loop (A).

Product maintenance - washing instructions:

Machine wash at 40°C. Do not bleach. Do not iron. Do not dry-clean. Do not tumble dry. Dry on flat surface. Do not hang out.

Maintain the velcro free of dirt preventing them from threads, fluffs, etc. to keep its characteristics. Close all the velcro straps on the brace surface before washing the product.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.