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Read these instructions carefully before using the product. If you have any questions, consult your doctor or the shop where you purchased it.

**INDICATIONS**

Post-operative cardiac and thoracic surgery. It can even be used in ICUs.

**INSTRUCTIONS FOR USE**

- Place the straps (A) over the patient's shoulders to leave the thoracic protector's adjustable band (C) at the height of the armpits.
- Fit the thoracic protector, ensuring that the two handles (D) cannot touch even with the most violent of coughs (separate by a distance of between 5 and 10 cm). Close the pretensioners (B), making sure that the distance is maintained.
- The crocodile-type Velcro® fastener (E) on the adjustable band (C) can be moved, enabling it to be removed to fit the circumference of the patient.
- Adjust the height of the straps to the required height of the patient. Once the ECOPRIM3 has been fitted to the patient, if necessary, remove the crocodile-type Velcro® fasteners (E) from each end of the straps (A) and the adjustable band (C) and trim any excess off the straps to the desired height and the two sides of the adjustable bands (C) equally. Lastly, reattach the crocodile-type Velcro® fasteners (E) at each corresponding end.
- To use, bring the thoracic protector's handles (D) together with one hand, exerting appropriate pressure on the thoracic circumference.
- The thoracic protector must be worn permanently (when going to bed, getting up, expectorating, etc.).
- It is advisable for the patient to sleep with the thoracic protector fitted.
- Correct use of the thoracic protector will provide safety and comfort.
- If you have any questions about its use, consult the specialist professional.
- Its use is recommended for at least a month after surgery.

⚠ Failure to observe these precautions could lead to situations of risk and injury.

- Wear the thoracic protector over a T-shirt or hospital clothes. It should only be used by a single patient.

**PRECAUTIONS**

- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment. Only a doctor or health professional is trained to perform the fitting of the product.
- For maximum effectiveness, observe the daily usage instructions of your doctor or the health professional. To ensure the full therapeutic effect, the product must be correctly adjusted.
- An orthosis for personal, non-transferable use. It must not be used by another patient or for any other purpose than that prescribed. Do not let children play with this device.
- If any side effects are experienced, your doctor should immediately be informed. You must inform your doctor if you suffer from any skin conditions. Avoid using topical creams that come into contact with the orthosis during treatment.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Immediately remove the orthosis in the event of evident irritation, pressure, pain or discomfort. Do not continue use until skin integrity and comfort are assured.
- Regularly check that it remains in good condition for use.

**MAINTENANCE INSTRUCTIONS**

Hand wash. Do not use bleach. Do not iron. Do not dry clean. Do not tumble dry. Dry on a flat surface. Do not hang up. Keep the Velcro® fasteners free from fluff or lint to maintain their adhesive properties. Attach all of the Velcro® fasteners to the surface of the orthosis before washing it.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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