ES

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ENG

Indications:

These orthoses are indicated for inflammatory and posttraumatic processes of the carpal region, carpal tunnel syndrome and tenosynovitis.

Preliminary observations:

- The doctor is the only person who can determine the duration of the treatment.
- You must follow the doctor's instructions regarding the daily duration of use.
- The initial fitting should be conducted by your doctor, or by an orthopaedic technician if indicated by your doctor.
- The use of topical medication must be supervised by your medical practitioner.
- Where pressure is applied, the skin should not be damaged or hypersensitive.
- Your orthopaedic technician may shape the product by applying local heat for better fitting.
- If the padding or closure systems are not mounted, they should be adjusted according to the anatomy and/or pathology of the patient.

Precautions:

- For full effectiveness, you should use the correct size (and side) of the product on the patient.
- The orthosis requires prior assembly and fitting by an orthopaedic technician or a qualified professional. After the fitting, the orthosis must not be modified except by qualified professional.
- The orthosis is for a single patient. Do not reuse it with other patients.
- You should always follow the general and specific instructions given by the doctor.
- Treatment should be reviewed frequently by the doctor. If you notice any side effects, you should inform your doctor immediately.
- You should inform your doctor if you have any skin conditions or if any type of sensitivity appears.
- The doctor should consider the use of topical creams with the orthosis. The correct use of the orthosis depends on the condition of its components. It must be checked periodically.
- Check that there is no breakage, wear and tear, etc.
- It is advisable to keep the Velcro closures clean by removing any threads, lint, etc. that may have stuck to them.
- Regularly clean the product and maintain good personal hygiene.
- The healthcare professional who monitors the patient's treatment may indicate the suitability of the product or suggest replacing it if the product is damaged or worn out.

Washing instructions:

- Clean the product surfaces with a cloth dampened in soapy water to eliminate dirt.
- Keep the Velcro straps clean by removing any threads, lint, etc. to preserve their adhesive properties.
- Close all Velcro straps before washing the product.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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