ES

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ENG

The orthosis 28542P is a lightweight splint made in low temperature thermoplastic, designed to stabilize the wrist at night.

Indications

Carpal Tunnel Syndrome. The brace keeps the wrist in neutral position avoiding movements of the wrist while sleeping.

Cautions

- The doctor is the person qualified to prescribe and decide the duration of the treatment. The orthopaedic technician is the person prepared to fit the product to the forearm, wrist and hand. For a correct function, you should follow his indications and the instructions of use.
- For a complete therapeutic treatment, you should use the product correctly adjusted and in its appropriate size.
- On the pressure points, the skin should not be injured nor irritated. If you notice any side effects, you should immediately contact your doctor. The doctor should be informed if the patient suffers of skin affection.
- You should avoid the use of topic creams in contact with the orthosis during the treatment.
- The correct use of the brace depends on the state of its components (make sure there is no breakage, no damage, etc.) so it should be checked periodically by your doctor or healthcare professional, who may indicate the suitability of the product or the convenience to replace it if the product is damaged.
- This brace is personal and non-transferable. Do not use it for a different use than the one which it has been prescribed.
- Do not allow children to play with this brace.
- It is advisable to take care of the cleaning of the quick release Velcros closures to keep its characteristics, removing threads, fluffs, etc. Clean the product regularly and maintain good personal hygiene.

Warnings

- Make sure that the straps are not too tight.
- Check your skin daily, if you see any superficial irritation signs, remove immediately the brace.
- This product is made of Kydex and is latex free.
- Keep the brace away from fire.

Washing instructions

- The plastic part of the brace can be washed with a wet cloth with soapy water, then rinse carefully.
- The straps can be machine washed at 30°. Do not bleach. Do not tumble dry. Do not iron. Do not dry-clean. Dry flat.

Presentation

Available in 2 sizes. Size S: 13-17 cm wrist circumference, Size L: 18-22 cm wrist circumference.

How to use

- Place the brace on the wrist, make sure the support is complete and not annoying (FIG. 1).
- Close the middle strap around the wrist (FIG. 2).
- Then close the distal strap over the knuckles.
- Finally, close the upper strap (FIG. 3).
- If necessary, re-adjust the straps in order to feel comfort. (FIG. 4).

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PRIM, S.A.