FR

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ENG

Indications: Joint pain, bursitis, tendinitis, arthritis, osteoarthritis, patellar injuries, lateral knee ligament injuries, Osgood-Schlatter disease, chondromalacia patella, patellaer subluxation, post injury and post-surgical treatment.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you observe any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

Instructions for proper fitting:

All items feature fastening straps, a blue dot for the proximal area (thigh) and a red dot for the distal area. All models are available in 5 sizes. (S, M, L, XL, XXL).
T130 Closed knee brace with monocentric joint and **T140 Closed knee brace with polycentric joint:**

Fig.1: Open the Velcro fasteners and fit the brace to the leg. Position the knee brace ensuring that the opening is over the kneecap (A) and centring the joint on the axis of the knee (B). The joint should enable the knee to flex.

Fig.2: Then close the top strap marked with the blue dot (C).
Fig.3: Repeat the operation with the bottom strap marked with the red dot (D).
Fig.4: To remove the joint, pull the black tab (E). This will reveal the hinged joint. Fold the knee brace and pull out the joint. To insert the joint, follow the above steps in reverse.

The joint should be adjusted by bending the bars and always using the right tool (Allen key). The joint must be made according to the patient's anatomy. Do not handle the inside of the joint where the hinge is located.

Product maintenance - washing instructions:

This product is latex free. Wash at 40°C. Do not dry clean. Do not iron. Do not use bleach. Do not tumble dry.

Models T130 and T140 must have their joints removed before washing. The joints should be wiped with a damp cloth. Do not grease the hinges.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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