FR

Indications: Fractures vertébrales.

Précautions:

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Indications: Vertebral fractures.

Precautions:

- Read these instructions carefully before using the orthosis. If you need more information, consult your doctor, orthopaedic technician or qualified personnel.
- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment. The orthopaedic technician is the person qualified to put together the splints, properly configure the hyperextension brace and carry out the fitting of the product.
- For maximum effectiveness, observe the daily usage instructions of your doctor, orthopaedic technician or qualified personnel.
- To ensure the full therapeutic effect, you must use the correct size.
- Orthosis for personal, non-transferable use. It should not be used by another patient.
- This orthosis must not be used for any other purpose than that prescribed. Do not let children play with this device.
- You must inform your doctor if you suffer from any skin conditions. Avoid using topical creams that come into contact with the orthosis during treatment.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Do not wear the support belt while having a bath, shower, etc.
- For reasons of hygiene and comfort, it is advisable to wear a cotton garment under the device.
- Regularly check that the belt and/or tensioners are tight and in good condition (free of grease, tears, etc.).
- Periodically check that the screws have not lost their initial tightness. If they have, consult your doctor, orthopaedic technician or qualified personnel for tightening.

Instructions for proper fitting:

The patient may need the help of another person to fit this orthosis. It is advisable to use a mirror to fit the hypertension brace. Ensure that the tensioning closure is in the open position.

Fig.1: Position the brace on the patient observing the direction of the arrow at the top of the corset.

Fig.2: Pass the lumbar plate around the back holding the belt with one hand and the brace positioned on the body with the other. The help of a second person is advisable to perform this procedure.

Fig.3: Without releasing the brace, use the thumb of the hand holding the belt to pull the blue cord of the plastic securing buckle and connect the buckle to its housing.

Fig.4: Finish positioning the brace correctly with both hands and tighten the orthosis by turning the locking lever to its closed position.

Fig.5: It is possible to adjust the angle of the sternal support (by increments of 6°).

(Only for the C35+ Avant brace).

Fig.6: Locking and unlocking the pelvic band: (Only for the C35+ Avant, C35 Basic Avant braces and C35 Basic Slim).

6.1 To lock, unscrew screw (B) to remove the cover.

6.2 The brace is supplied without screw (A), with the pelvic band in the free movement position.

6.3 Position locking screw (A), supplied in the accessories bag (bag with two screws).

6.4 Reattach the cover by tightening screw (B).

Product maintenance - washing instructions:

Metal structure and fixed padding:
Wash the entire metal structure and non-removable pads (pelvic band and side plates) with a chamois leather and warm water. Dry with a dry cloth and do not use bleach.

*Removable textile parts:

The lumbar, pectoral and pubic pads (C34 Avant) are removable for cleaning. Wash in cold water. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.
BEWARE: If an incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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