IT

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ENG

Indications:
PS100: Post-operative of the foot and toes. Feet with immobilising bandages or plaster casts. Oedema of the foot when a conventional shoe cannot be worn.

PS200: Post-operative of the forefoot and toes. Feet with immobilising bandages or plaster casts.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Treatment should be regularly monitored by a doctor.
- If any side effects are experienced, your doctor should immediately be informed.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath or shower, or sleeping at night.

Istruzioni for proper fitting:

Select the appropriate size taking into account the patient's shoe size and post-operative bandage.

- Insert the foot into the shoe, making sure that the heel is correctly positioned in the heel reinforcement.
- Then cover the instep and fasten the closure to obtain a secure fit.
- Adjust the closure to achieve an optimum degree of compression.
- The ends of the toes should not protrude over the front edge of the shoe. There should be a gap of a few centimetres.

Product maintenance - washing instructions:

Close all the Velcro straps. Hand wash with cold water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Rinse away all remnants of soap to prevent possible skin irritation and damage to the shoe. Dry at room temperature. Avoid exposure to direct heat sources.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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Post-operative instructions:
* Post-operative instructions:
* Post-operative instructions:
* Post-operative instructions:

Precautions:
* Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
* Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
* Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
* The general instructions of use and specific indications determined by the doctor should always be followed.
* Treatment should be regularly monitored by a doctor.
* If any side effects are experienced, your doctor should immediately be informed.
* Your doctor might consider the use of topical creams in conjunction with the orthosis.
* In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
* For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
* Do not let children play with this device.
* Clean the product regularly and maintain proper hygiene.
* The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
* Do not wear the orthosis while having a bath or shower, or sleeping at night.

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PRIM, S.A.