FR

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ENG

Indications: Post-surgical or post-traumatic knee where flexion/extension control may be necessary, for example, after lateral or cruciate ligament surgery. Also indicated for cases of pre-rigorous immobilisation.

Precautions:

- Use these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.

- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
- The orthosis requires assembly and prior adjustment by qualified personnel. After the orthosis has been adapted to a patient, it should only be modified by qualified personnel.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by your doctor should always be followed.
- Treatment should be regularly monitored by a doctor.
- If any side effects are experienced, your doctor should immediately be informed.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- For hygiene and comfort, you are recommended to wear a cotton garment, preferably seamless, under the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath, shower, etc.
- Regularly check that the straps are tense and in good condition (free of grease, cuts, tears, etc.). Make sure that the Velcro fasteners are free from fluff and lint to ensure the rigid surfaces with a damp cloth.

The flexion and extension stops on the inner and outer joints should have the same settings. The orthosis can only be used if the locks are correctly positioned. Likewise, the telescopic mechanism should be equally configured on both sides and properly adjusted. In the event of malfunction of any of the locking mechanisms, immediately contact your orthotist or a specialist technician or the shop where you purchased the orthosis.

Instructions for proper fitting:
Length (telescopic device):

A. Release all of the straps and position the central axis of the joint over the axis of the knee. To adjust the length of the straps of the orthosis, press the buckle releases and lengthen or shorten the straps until the required length is reached.

B. Fasten the buckles and pull the end of the straps as shown in figure B. If the strap is too long, trim the excess. The crocodile-style system allows the end of the strap to be fastened and kept out of the way.

Limitation of joint flexion/extension (articulation):

C. Press the blue switch to the "UNLOCK" position to unlock the joint at 0°, 15° and 30°, and to the "LOCK" position to lock them. Follow the prescriber's instructions if necessary.

D. Move the flexion/extension position selectors to the angles indicated by the prescriber.

E. To release the straps, pull on the buckle in the direction indicated in image E. Once the tab comes out of the housing, the buckle is released.

Product maintenance - washing instructions:

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Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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