mottio^{HF}
sport range

T129

Cincha patelar

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ENG

Indications:

Treatment of Osgood-Schlatter disease. Treatment of patellar tendinitis (jumper's knee). Treatment of patellar chondromalacia.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only a doctor is qualified to prescribe and determine the duration of treatment.
- The orthosis requires assembly and prior adjustment by qualified personnel.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Treatment should be regularly checked by qualified personnel.
- If you experience any side effects, you should immediately inform your doctor.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Clean the product regularly and maintain proper hygiene.

ENG

- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath or shower, or sleeping at night.

Instructions for proper fitting:

- Detach one of the parts of the adjustment strap, leaving it completely free.
- Fit the strap to the leg at the height of the patellar tendon (just below the patella) and centre it.
- Pass the strap through the buckle again, pull both straps backwards around the leg and fasten.
- Stand up and walk around to check that the compression is correct, loosening or tightening the fastening straps as required.
- Adjust the length of the straps if necessary by trimming the ends. Only trim along the two flat arrow-shaped areas between the Velcros (see Fig. 1).

Product maintenance - washing instructions:

Before washing the orthosis, fasten the Velcros to the strap and remove the silicone pad. Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

To dry the orthosis, use a towel to absorb as much moisture as possible and then leave to dry at room temperature. Do not hang up, iron or expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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