## FR

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- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The orthosis requires assembly and prior adjustment by qualified personnel.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by your doctor should always be followed.
- Treatment with the orthosis should be regularly monitored by the doctor.
- If any side effects are experienced, your doctor should immediately be informed.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Make sure that the Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath or shower, or sleeping at night.

**Instructions for proper fitting:**

- Select the right size by measuring the patient's wrist circumference. Excessive compression can cause discomfort. We recommend adjusting the compression until a comfortable fit is achieved.

The palmar and thumb splints are supplied pre-formed but may require adjustment to fit the patient's anatomy.

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**Product maintenance - washing instructions:**
Before washing the support, fasten the Velcros and remove the palmar and thumb splints, which should be cleaned with a damp cloth. Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

## ENG

**Indications:**

**Wrist supports with palmar splint (C500, C600):**

Carpal tunnel syndrome, Guyon's canal syndrome, De Quervain syndrome, tendinitis, tenosynovitis, ligament injuries, trauma, post-surgery, wrist and sports injury stabiliser.

**Wrist and thumb supports without palmar splint (C400, C700):**

Rhizarthrosis, Bennett's fracture dislocation, tendinitis, tenosynovitis, ligament injuries, trauma, post-surgery, sports injuries, metacarpophalangeal and trapeziometacarpal joint stabiliser.

**Wrist and thumb supports with palmar splint (C800):**

Carpal tunnel syndrome, Guyon's canal syndrome, De Quervain syndrome, tendinitis, tenosynovitis, ligament injuries, trauma, post-surgery, wrist stabiliser, sports injuries. Rhizarthrosis, Bennett's fracture dislocation, metacarpophalangeal and trapeziometacarpal joint stabiliser.

**Precautions:**

- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The orthosis requires assembly and prior adjustment by qualified personnel.
- Although the orthosis is not for single use, it should only be used by one patient.
- The product should not be reused by other patients.
- The general instructions of use and specific indications determined by your doctor should always be followed.
- Treatment with the orthosis should be regularly monitored by the doctor.
- If any side effects are experienced, your doctor should immediately be informed.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Make sure that the Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath or shower, or sleeping at night.

**Instructions for proper fitting:**

- Select the right size by measuring the patient's wrist circumference. Excessive compression can cause discomfort. We recommend adjusting the compression until a comfortable fit is achieved.

The palmar and thumb splints are supplied pre-formed but may require adjustment to fit the patient's anatomy.

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