HÄNDELEDDSORTOSE

28210P

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HÄNDLEDSORTOS

28210P

INDICATIONS: Helping the patient to limit movements that may cause injury or pain.

CAUTIONS:

For a correct function, you should use the appropriate size and side of the product to fit to the patient.

The brace requires an assembly and a previous adaptation performed by an orthopaedic technician or qualified professional. Once fitted to the patient, the orthosis should not be modified except by qualified professional.

The brace is for a single patient. Do not reuse in other patients. You should always respect the general instructions of use and the specific indications prescribed by the doctor.

The follow-up should be reviewed frequently by the doctor. If you notice any side effects, you must contact immediately your doctor. The doctor should be informed if the patient suffers of skin affection or any type of skin sensitization.

The doctor should consider the possible use of topic creams together with the use of the brace.

On the pressure points, the skin should not be injured nor irritated.

The correct use of the brace depends on the state of its components, so it should be checked periodically.

Make sure there is no breakage, no damage, etc. on the brace. It is advisable to take care of the cleaning of the quick release Velcro closures to keep its characteristics, removing threads, fluffs, etc.

Clean the product regularly and maintain good personal hygiene.

The medical staff following the patient's treatment may indicate the suitability of the product or the convenience to replace it if the product is damaged.

CARE INSTRUCTIONS:

Before washing the brace, remove the stays (the ones with open lodging). Hand washable, lukewarm water. Do not bleach. Do not iron. Do not dry-clean. Do not tumble dry. Dry on flat surface. Do not hang-dry. It is advisable to maintain the velcros free of dirt preventing them from threads, fluffs, etc. In order to keep their adhesion characteristics. Close all the velcros straps on the brace surface before washing the product.

NOTE: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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