MPK100 • MPK101 • MPK102

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·
Tilpasning instruktion · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions ·
Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·
Istruzioni per l'adattamento · Uždėjimo instrukcijos · Pielāgošanas instrukcijas · Montage-instructies
· Tilpassingsinstruksjon · Instrukcja obsługi · Instruções de adaptação · Instructiuni de reglare ·
Инструкции по адаптации · Návod na použitie · Utprovningsinstruktion

ES

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ENG

Description and intended use:

MPK100: Highly flexible wrist brace design that stabilises by wrapping around the thumb and wrist, exerting the necessary compression to hold the joint and relieve pain, while maintaining the functionality of the hand.

To achieve this, it is made from a breathable, elastic material that fits perfectly to the hand and contains foam for increased padding and comfort.

MPK101: Immobilising wrist brace made from padded, breathable, elastic fabric to provide a snug fit. The brace achieves immobilisation thanks to its internal malleable aluminium palmar splint, which immobilises the wrist joint, and its dorsal splint, which provides the necessary rigidity for better stabilisation of the joint. The foam and its ergonomic design provide the hand with total comfort.

MPK102: Thumb attachment for the immobilising wrist brace made from the same materials and containing an aluminium splint to immobilise the thumb joint as required by the injury. This splint is mouldable and can fit the hand of each patient.

Indications:

MPK100: Treatment for tendinitis, sprains, mild joint injury, minor instability, sports injury prevention.

MPK101: Immobilisation treatment after surgery or injury; immobilisation and protection of the wrist joint (carpometacarpal joint): support for the wrist in cases of muscular dystrophy.

MPK102: Immobilisation and protection of the thumb joint (metacarpophalangeal joint).

Precautions:

ENG

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

Instructions for proper fitting:

MPK100: Once the correct size has been chosen for the patient, the thinner end, rolled up on itself, should be fitted and fastened with the velcro and velour, forming an opening the same size as the thumb (Fig. 1). Next, place on the thumb adjusting the size of the opening if necessary (Fig. 2), and then wrap around the wrist with the necessary pressure for optimum compression, without immobilising (Fig. 3). Lastly, close the orthosis by fastening the velcro to the velour (Fig. 4).

MPK101: Once the correct size and side have been chosen for the patient, the hand should be inserted into the wrist brace; if necessary, it is possible to adjust the palmar splint to the needs of the patient. Lastly, close the velcro fasteners so that the brace fits correctly.

MPK102: This product comes in one size. For fitting, open the wrist brace, place face down (Fig. 1), insert the attachment through the thumb opening, as shown in (Fig. 2), as far as necessary depending on the length of the patient's thumb (Fig. 3) and fasten as shown in (Fig. 4). Then close the orthosis normally, as indicated in the instructions for MPK101.

Product maintenance - washing instructions:

Before washing, close all of the velcros. Hand wash with warm water (max. 30°C). Use neutral detergent. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

For drying, use a towel to absorb as much moisture as possible and then leave to dry at room temperature. If the orthosis is not well rinsed out, detergent residue can remain and cause irritation to the skin and damage to the product. Rinse the orthosis with plenty of water to remove any remaining detergent which could damage the product or cause skin irritation.

Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc. Do not use alcohol, creams or solvents for cleaning.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.