MPK700 • MPK701 • MPK800

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·  
Tilpasning instruktión · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions ·  
Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·  
Istruzioni per l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi ·  
Instruções de adaptação · Instruțiuni de reglare · Инструкции по адаптации ·  
Návod na použitie · Utprovningsinstruktion · Navodila za prilagoditev · Uždejimo instrukcijos ·  
Pielāgošanas instrukcijas

ES

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## ENG

### Description and intended use:

**MPK700:** Knee support made from breathable, elastic and padded fabric, providing a fit suited to the needs of each patient. It features lateral stays to stabilise the knee joint in the mediolateral direction, allowing movement, and a removable patella pad to centre and immobilise the knee if necessary. Size 2 also features a silicone patella pad. Fastening is by means of two velcro straps, which can also regulate the compression, leaving the knee pit free.

**MPK701:** The immobilising knee orthosis is made from breathable honeycomb material with terrycloth interior. The product immobilises thanks to three splints: two side and one back, all of them ergonomically shaped, which fix the leg in extension.

The fastening system consists of four velcros located on the front, enabling the kneecap to remain free and providing greater comfort.

**MPK800:** Ankle support made from breathable, elastic and padded fabric. Its front opening provides an optimum fit and greater comfort during walking. Fastening is by means of a velcro system that enables adjustment and regulation of compression according to the needs of the patient. Stabilisation of the ankle is achieved thanks to its crossover strap design which provides better mediolateral control.

### Indications:

**MPK700:** Slight contusions, femoropatellar injuries. Post-surgical treatments. Slight instabilities of the knee joint. During the rehabilitation process. Any knee condition that requires centring or unloading of the patella. Osgood Schlatter disease.

**MPK701:** Disorders that require immobilisation of the knee joint to enable treatment; pre and post-surgical treatment of the knee; after injuries or fractures.

**MPK800:** Treatment of mild sprains, tendonitis and recurrent injuries.

**Contraindications:** Contraindicated for open wounds with swelling, redness and heat accumulation.

### Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

### Instructions for proper fitting:

**MPK700:** Once the appropriate size for the patient has been chosen, fit the orthosis to the knee, making sure that the opening in the front part of the product is positioned over the kneecap. Then fasten the velcros located on the back, tightening them to achieve correct stabilisation of the knee joint.

It includes a removable patella pad for stabilisation of the kneecap (Fig. 1).

Size 2 features a silicone pad attachment that exerts pressure on the patellar tendon, relieving pain. The silicone pad can be fitted by placing it on the patella pad as shown in (Fig. 2) if both attachments are needed, or, by attaching it directly to the knee support as shown in (Fig. 3) if only the silicone pad is needed.

## ENG

**MPK701:** Once the appropriate size for the patient has been chosen, fit the orthosis from the back, making sure that the front fasteners overlap so that the knee is free in the central area. Fasten using the velcros at the front, ensuring that the knee is correctly immobilised.

**MPK800:** Once the appropriate size has been chosen, fit the product to the patient's foot (Fig. 1) and fasten the upper velcro to the front area of the ankle (Fig. 2). Then cross over and fasten the side straps to stabilise the ankle (Fig. 3).

### Product maintenance - washing instructions:

Before washing, close all of the velcros. Hand wash with warm water (max. 30°C). Use neutral detergent. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

For drying, use a towel to absorb as much moisture as possible and then leave to dry at room temperature. If the orthosis is not well rinsed out, detergent residue can remain and cause irritation to the skin and damage to the product. Rinse the orthosis with plenty of water to remove any remaining detergent which could damage the product or cause skin irritation.

Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc. Do not use alcohol, creams or solvents for cleaning.

**NOTE:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.