# Humerux

EVOH300

Ortesis de Húmero

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## ENG

**Description and intended use:** Thermoformed bi-valve humerus orthosis made of high-density polyethylene, covered in a soft padded textile cover. Consists of 2 thermoplastic plates assembled in a breathable honeycomb fabric lining and held together by fabric and webbing. Adjustment system via 3 humeral velour straps threaded on 3 buckles with micro-hook fastening. Suspension and fastening system via chest strap. The orthosis comes reinforced with a clamping strap (Y-traction System) which allows the tension that is exerted to be increased and adjusted, thus achieving better fastening and support of the humerus.

**Indications:** Diaphyseal humerus fractures, post-surgical fracture treatment and stabilisation of the fracture site during the consolidation process.

**Contraindications:**

- Do not use on open wounds with swelling.
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

**Precautions:**

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.
- Do not let children play with this device.

**Instructions for proper fitting:**

1. Select the right size for your body dimensions. Upper arm contour.
2. Open the arm and chest support and adjustment straps, leaving them in their maximum open position (fig. 1).
3. Insert the affected arm into the orthosis and pull it into the correct position, making sure that the humeral cup correctly lifts the shoulder (fig. 2).
4. Adjust the arm straps until you achieve the desired support and pressure from both plates (fig. 3).
5. Ensure that the underarm protection pad is correctly positioned under the armpit (fig. 2).
6. Adjust the chest strap by adjusting the micro-hook fasteners (fig. 4).

**In case it is necessary to adjust the thermoplastic plates:**

1. Remove them from their fabric housing.
2. Heat the plates with a heat gun to about 150°C at a distance of 15cm.
3. Shape according to the patient's anatomy.
4. Re-insert the plates into their fabric housing.

**Product maintenance - washing instructions:** Before washing, remove the plates and close all of the Velcro straps. Wipe the plates down with a damp cloth. Wash at 40°C. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.