# Elcross

L · U · X resistex  
CARBON

3131BG • 3132BG • 3132G • 3133BG • 3133G • 3134BG  
3138BG

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·  
Tilpasning instruktion · Οδηγίες εφαρμογής · Fitting instructions · Sovitusohjeet ·  
Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások · Istruzioni  
per l'adattamento · Montage-instructies · Tilpassingsinstruksjon · Instrukcja obsługi  
· Instruções de adaptação · Instructiuni de reglare · Инструкции по адаптации ·  
Návod na použitie · Utprovningsinstruktion

# ES

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## ENG

### Description and intended use:

**Back support belts:** Made from an elastic and soft material that adapts 100% to the body with carbon fibre material on its inner side. Comfortable and convenient, they have two paravertebral steel stays that can be removed and are reinforced with elastic tensioners that provide a high level of support, ideal for any process that requires torso immobilisation at the sacrolumbar level.

**Back support:** Made from an elastic and soft material that adapts 100% to the body with carbon fibre material on its inner side. Comfortable and convenient. Thanks to its straps reinforced with elastic that increases traction, it pulls the shoulders back being very effective for correct posture and for back pain.

### Indications:

**Elcross Lux Back Support Belts 3131BG, 3132BG, 3132G, 3133BG, 3133G, 3134BG:** Lower back pain, lumbosciatica, disc herniation, muscle weakness or atony, postoperative lumbar processes and degenerative processes of the spine such as arthritis or lumbar arthrosis.

**Elcross Lux back support 3138BG:** Posture reminder. Upper back pain.

### Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

### Instructions for proper fitting:

It is advisable to use a mirror when putting the belt on. For hygiene and comfort, you are recommended to wear a cotton garment, preferably seamless, underneath the belt. Elcross Lux breathable belts are provided with front or side Velcro straps

(depending on the model) (A) ergonomically designed, which will help you put on and take off the belt. They also have adjustment straps (B) which increase the support and immobilisation of the lumbosacral region which are fitted over the main fastening. The design of the belt features Velcro fasteners (C) to hold the straps in place during handling and prevent them from sticking to each other or to the body of the belt.

**Elcross Lux lumbosacral belt model 3131BG, 3132BG, 3132G, 3133BG, 3133G (Fig. 1):**

- Make sure that the adjustment straps (B) are placed over the Velcro fasteners (C).
- Detach the Velcro strap (A) on the front of the belt. Place the Elcross Lux belt at the correct height and position, (3 cm above the buttock crease), without applying any tension.
- Adjust the belt correctly and close the Velcro strap on the front of the belt (A). Detach the side straps (B) and pull them with equivalent tension, from the back to the front. Close the side straps (B) over the front strap (A).

**Elcross Lux abdominal pendulum belt model 3134BG (Fig. 2):**

- Make sure that the adjustment straps (B) are placed over the Velcro fasteners (C).
- Detach the Velcro strap (A) from side of the belt. Place the Elcross Lux belt at the correct height and position, (3 cm above the buttock crease), without applying any tension. Since this is a belt for pendulous abdomen, it will be placed with the Velcro fastener (A) on the side, since the front part serves to accommodate the abdomen.
- Adjust the belt correctly and close the Velcro strap on the side of the belt (A). Detach the adjustment straps (B) and pull them with equivalent tension, from the back to the front. Close the straps (B) on the front of the belt, making sure that they remain underneath the belly, attached to the Velcro straps.

**Elcross Lux back support 3138BG (Fig. 3):**

- It is advisable to use a mirror when putting the back support on. For hygiene and comfort, it is recommended to wear a cotton garment, preferably seamless, under the product.
- Wear the appropriate size back support as a waistcoat and close the Velcro to fit the patient's body shape.

**Product maintenance - washing instructions:**

Machine wash at 40°C. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Close all the Velcro straps on the surface of the garment before washing.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.