Primospine
CARE CARE+

PRS610BG
PRS610G
PRS620BG
PRS620G

Faja lumbosacra semirrígida

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ENG

Description and intended use:

Made from soft material that fits your body perfectly. Comfortable, lightweight and snug. Applies light compression in the case of PRS610BG/PRS610G and moderate compression in the case of PRS620BG/PRS620G, suitable for mild and/or moderate pain or for long-term treatments.

Indications:

PRS610BG/PRS610G: Mild lower back pain, mild post injury, prophylaxis in sporting activities and muscle weakness.

PRS620BG/PRS620G: Moderate low back pain, lumbosciatica, disc herniation, degenerative processes of the lumbar spine, rehabilitation, muscle weakness.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- For the orthosis to perform its function, the right size for the patient must be used. If the patient is close to the maximum range of a size and has a protruding abdomen, choose the next size up.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.

ENG

- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Some materials of the product are not flame retardant, therefore it is recommended not to use it in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.
- The accessories are designed to be used exclusively with these support belt models.

Instructions for proper fitting:

The following aspects should be taken into account when fitting the product:

1. Place the brace around the body so that the braces are centred on the back, taking the spine as the midpoint. The lower part of the belt should be at the level of the pelvis. In the case of PRS620BG/PRS620G the tensioning straps must remain loose during the fitting of the main part of the belt.
2. Insert your fingers into the push pin on the right-hand side of the front section.
3. Press the left side of the front over the abdomen and pull the right side until you can close the microhook of one front over the other.
4. For the PRS620BG/PRS620G model, pull both ends of the tensioning straps forward to increase the amount of compression.
5. Then fasten the ends of the tensioning straps over the front fastening of the belt.

We also offer the PRS6APL lumbar plate accessory, which can be attached to the PRS610BG/PRS610G belt model by fastening the microhook of the plate onto the velour oval that the belt has on the inside of the back section. The PRS620BG/PRS620G model has a built-in lumbar plate.

Note for the person fitting the belt:

- To fit the belt, position the paravertebral stays on both sides of the spine.
- Shape the stays to fit the anatomical curves of the patient.

Product maintenance - washing instructions:

Before washing, make sure that all of the Velcro fasteners are closed. Hand wash with warm water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.