**Prin**spine<sup>®</sup>  
MAMMY

**PRS615BG**  
**PRS615G**

Faja embarazada  
مشد الحامل  
Těhotenský bederní pás  
Stützgurt für Schwangere  
Graviditetsbælte  
Ζώνη εγκυμοσύνης  
Rasedusaegne bandaaž  
Maternity back support belt  
Tukivyy raskaana oleville  
Ceinture de grossesse  
Steznik za trudnice  
Hastartó pánt kismamáknak  
Fascia per gravidanza  
Néščiosios juosta  
Grūtnieču josta  
Zwangerschapsband  
Magebånd for gravide  
Pas ciążyowy  
Faixa para grávida  
Centură pentru gravide  
Бандаж для беременных  
Tehotenský pás  
Korsett för gravida

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·  
Tilpasning instruksjon · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions ·  
Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·  
Istruzioni per l'adattamento · Uždėjimo instrukcijos · Pielāgošanas instrukcijas · Montage-instructies  
· Tilpassingsinstruksjon · Instrukcja obsługi · Instruções de adaptação · Instrucțiuni de reglare ·  
Инструкции по адаптации · Návod na použitie · Utprovningsinstruktion

ES

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## ENG

### **Description and intended use:**

Made from soft material that fits your body perfectly. Comfortable, lightweight and snug. Applies a light compression and support that is suitable for treating mild pain and/or significant abdominal bulge during pregnancy.

### **Indications:**

Mild lower back pain during pregnancy. Recommended for large increases in abdominal volume or for the last months of pregnancy.

### **Precautions:**

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Some materials of the product are not flame retardant, therefore it is recommended not to use it in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.

### **Instructions for proper fitting:**

It is advisable to fit the support belt in front of a mirror. The belt consists of two parts: an elastic rear section and a rigid front section in the shape of a concave band.

- FIG.1: The front section features symmetrical marks in the shape of small holes (A), which serve as a reference to ensure that the belt is centred.
- FIG.2: Place the front band (B) centred below the abdomen and hold with your hands. When fitting for the first time, it is advisable to have the help of another person to ensure correct placement.
- FIG.3: Firstly attach the tensioning straps to the velour fabric of the rear section.
- FIG.4: Then position the elastic section on the back and fasten to the front with the Velcro fasteners. Use the reference marks (A - Fig.1) to centre the belt in such a way that the left and right Velcro fasteners are positioned symmetrically.
- FIG.5: Close the rear section over one end. Repeat the action over the other end, making sure that the support belt is centred.
- FIG.6: Finally, adjust the belt's degree of compression by pulling the left and right tensioning straps at the same time (C).
- FIG.7: The tensioning straps (C) must always be attached to the front band (B).

### **Product maintenance - washing instructions:**

Before washing, make sure that all of the Velcro fasteners are closed. Hand wash with warm water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.