Prim spine
LABORAL LABORAL+

PRS625N
PRS635N
PRS6ATR

Fajas laborales
مشدات العمل
Pracovní bederní pásy
Arbeitsstützgurte
Støttebælter til arbejdsbrug
Ζώνες εργασίας
Töötugivööd
Industrial support belts
Tukivöyt työkäyttöön
Ceintures de travail
Darbo juostos
Darba jostas
Radni steznici
Munkavédelmi deréktámasztó övek
Fascia lavoro
Werkbanden
Arbeidsbelter
Pasy wysiłkowe
Faixas laborais
Brăuri pentru lucru
Бандажи для работы
Pracovné pásy
Korsetter för tunga lyft

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung ·
Tilpasning instruktion · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions ·
Sovitusohjeet · Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·
Istruzioni per l'adattamento · Uždejimo instrukcijos · Pielāgošanas instrukcijas · Montage-instructies
· Tilpassingsinstruksjon · Instrukcja obsługi · Instruções de adaptação · Instrucțiuni de reglare ·
Инструкции по адаптации · Návod na použitie · Utprovningsinstruktion

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ENG

Description and intended use:

Made from soft material that fits your body perfectly. Provides strong support and lumbosacral compression that serves as an effective protection against industrial injuries.

Indications:

Lumbago, lumbosciatica, intermittent overloading of the lumbar area. Repetitive movements. Efforts. Prevention of occupational injuries.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- For the orthosis to perform its function, the right size for the patient must be used. If the patient is close to the maximum range of a size and has a protruding abdomen, choose the next size up.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Some materials of the product are not flame retardant, therefore it is recommended not to use it in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.
- The accessories are designed to be used exclusively with these support belt models.

Instructions for proper fitting:

The following aspects should be taken into account when fitting the product:

1. Place the brace around the body so that the braces are centred on the back, taking the spine as the midpoint. The lower part of the belt should be at the level of the pelvis.
2. Insert your fingers into the push pin on the right-hand side of the front section.
3. Press the left side of the front over the abdomen and pull the right side until you can close the microhook of one front over the other.
4. Then pull both ends of the tensioning straps forward to increase the amount of compression.
5. Then fasten the ends of the tensioning straps over the front fastening of the belt.

In the case of the PRS625N belt, we offer the PRS6ATR accessory straps, which allow the user to loosen the belt without it coming off. The straps should be positioned as follows:

1. Fasten the straps to the belt with the help of the microhook fasteners by fastening them over the stays on the back of the belt and on the fastening on the front of the belt.
2. Fit the back support and adjust the length of the straps.

ENG

Note for the person fitting the belt:

- To fit the belt, position the paravertebral stays on both sides of the spine.
- Shape the stays to fit the anatomical curves of the patient.

Product maintenance - washing instructions:

Before washing, make sure that all of the Velcro fasteners are closed. Hand wash with warm water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.