## Primospine<sup>®</sup> MODULAR

Faja lumbosacra semirígida doble banda tensora

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## ENG

### Description and intended use:

Made from soft material that fits your body perfectly. Comfortable, lightweight and snug. Offers medium/high support and stabilisation of the sacrolumbar area of the torso.

### Indications:

Severe lower back pain, lumbosciatica, degenerative processes, muscle atony and weakness, pre- and post-operative treatments, disc herniation, spondyloarthritis and lumbar spondylitis, osteomalacia. Strong sacrolumbar support.

### Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- For the orthosis to perform its function, the right size for the patient must be used. If the patient is close to the maximum range of a size and has a protruding abdomen, choose the next size up.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Some materials of the product are not flame retardant, therefore it is recommended not to use it in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain proper hygiene.
- Do not wear the orthosis while having a bath, shower, etc.
- Do not let children play with this device.
- The accessories are designed to be used exclusively with these support belt models.

### Instructions for proper fitting:

The following aspects should be taken into account when fitting the product:

1. Place the brace around the waist so that the stays are centred on the back, taking the spine as the midpoint. The tensioning straps must be at the top. The lower part of the belt should be at the level of the pelvis.
2. Insert your fingers into the push pins of the right and left side of the front fastener.
3. Press the left side of the front over the abdomen and pull the right side until you can close the microhook of one front over the other. Remove your fingers from the push pin of the left hand to be able to close them.
4. Pull both ends of the upper straps forward to increase the amount of compression.
5. Then fasten the ends of the upper straps over the front fastening of the belt.

### Note for the person fitting the belt:

- To fit the belt, position the paravertebral stays on both sides of the spine.
- Shape the stays to fit the anatomical curves of the patient.
- A lumbar plate is included with the product to provide additional support.
- Formable and lined polyethylene modules are available as an accessory for a more rigid support. Shoulder supports are also available as an accessory to provide further back support.

### Product maintenance - washing instructions:

Product maintenance - washing instructions: Hand wash with warm water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.