Prim spine

PRS670

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ENG

Description and intended use:

Brace that helps to hyperextend the spine in order to release the load on the vertebral bodies. It also limits the movements associated with flexo-extension and laterality.

Indications:

Suitable for fractures and vertebral crushing or pain caused by: traumatic injury, dorsal kyphosis, osteoporosis, Pott's disease, lumbago, lumbar sciatica, chronic back pain, post surgery for fractures of the vertebral bodies, or permanent orthotic care following surgery for tumours or metastasis of the vertebral bodies.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Only a doctor is qualified to prescribe and determine the duration of treatment, and carry out subsequent monitoring. The orthopaedic technician is the person qualified to position the splints, configure the hyperextension brace in line with the required shape, and fit the product correctly.
- For maximum effectiveness, follow the daily instructions for use provided by your doctor, orthopaedic technician or another qualified person.
- If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately. Discontinue use until assessed by a specialist.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- For the orthosis to function properly, its components must be regularly checked to make sure they are in good condition. We recommend checking the joints and moving parts to make sure they are working properly and fitted correctly (tightness, degree of correction, etc.). The medical staff responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
- Clean the product regularly and maintain proper hygiene.
- For reasons of hygiene and comfort, it is advisable to wear a cotton garment under the device..
- Do not let children play with this device.
- Check regularly to make sure the screws have not lost their original tightness. If they become loose, ask your doctor, orthopaedic technician or another qualified person to adjust them.

Instructions for correct fitting by the technician:

Select the size based on the measurements for trunk height and hip circumference in the iliac crest area. Adjust the width and height of the product using the splints, which slide behind one another when the screws are loosened. First, the side fastener must be undone. If additional height adjustment is required, this can be done using the chest plate, which has a basic adjustment function with three settings as well as a system offering more precise adjustment. To adjust the width

at the lower part of the brace, take out the central pad and put it back once the product has been adjusted. The brace allows you to adjust the angle of the sternum support by increments of 6°.

After you have adjusted the structure, you will need to adjust the rater belts, which must be long enough to allow the lumbar plate to produce correct hyperextension of the spine.

If the patient is short in stature, the plate has a surround that can be trimmed in order to remove excess material.

Instructions for correct fitting by the patient:

The patient may need the help of another person to fit this orthosis. You are advised to use a mirror when fitting the hypertension brace.

Make sure the tensioning fastener is in the open position.

Fig. 1: Position the brace on the patient, in accordance with the direction indicated by the arrow at the top.

Fig. 2: Move the lumbar plate to the back, holding the belt with one hand and the brace with the other.

Fig. 3: Without releasing the brace, insert the thumb of the hand holding the belt into the space in the plastic locking buckle, then connect the buckle to its housing.

Fig. 4: Tighten the orthosis by turning the locking lever until it reaches the closed position.

Fig. 5: The brace is supplied with the pelvic band in the free movement position. To lock the pelvic band, remove the cover by unscrewing the screw (B), then attach the locking screw (A) that is supplied in the bag of accessories. Reattach the cover by tightening the screw (B).

Subclavicular Attachment:

This accessory must be fitted by a qualified technician. This is a bilateral subclavicular support that replaces the sternal support and is indicated if the sternal area is not to be subjected to pressure.

For fitting, the screws marked in Fig. 6 must be removed. Then remove the sternal support Fig. 7 and fit the shoulder plates on both sides using one of the frame width adjustment screws Fig. 8. It should be noted that the plates have right and left sides and that their height should be adjusted according to the height of the patient.

Product maintenance - washing instructions:

The product has a cushioning system that can be completely removed for washing. Wash in cold water. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

A set of replacement padding is available in case of breakage or damage.

The structure of the brace should be cleaned using a damp cloth. You may use soap or detergent if necessary, as this will not affect the product.

Note:

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.