- 1850CH** ARNESES
- 1852CH** المراجع
- 1854CH** ÚPONY
- 1856CH** HARNESS
- 1858CH** HARNAIS
- 1860CH** SISTEMI DI RITENUTA
- 1865CH** ARNESES
- 1858AC** HAMURI
UCHYTENIE

ES

Estimado cliente,

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EN

Dear customer,

Thank you for trusting a Teyder product. You have purchased a quality product. Please, read the instructions carefully. If you have any doubt you can contact your doctor, orthopedist, or our customer service department.

We wish you have a quick recovery together with Teyder products.

QUALITY AND REGULATION

All our products have been manufactured following all CEE quality controls, and strictly following the European Regulation on Medical Devices – Regulation (EU) 2017/745 of the European Parliament of the Council.

SIZES

Selecting the correct size is important to guarantee the good product results. Please, consider the measurement charts presented on the product catalog to ensure that the size choice is correct.

INDICATIONS

Any restraint system is indicated to maintain the user's posture in the wheelchair, being the key to their quality of life. In general, the use of any of this range of products provides greater safety when moving, body stability and prevents slips and falls. Any of these products will be chosen according to the needs of postural control and support required by the user:

Harness: They are especially suitable for people with trunk instability. They allow complete support for the user, preventing the user from sliding off the seat (references 1850CH and 1852CH). In the case of the perineal harness (reference 1854CH), it is also indicated for patients with good anterolateral control with a tendency to slide off the seat.

Leg Abductor belt: These are the most suitable for users who need to keep both legs in permanent abduction (reference 1860CH).

Abdominal belt: Suitable for all types of people as they improve patient stability and safety (references 1856CH and 1858CH).

Wrist harness: Suitable for patients who need a complete support (reference 1865CH).

PRECAUTIONS

If while using the product you feel skin reaction, and allergy or skin irritation, please remove it quickly, and ask for medical advice.

Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the member state in which the user and/or patient are established.

WASHING INSTRUCTIONS

Wash by hand with a neutral soap. Do not wash it with liquids containing alcohols, bleach or dissolvent neither do not dry clean. To dry the product, do not expose it directly to heat sources (direct sun light, radiators, heaters, etc.). Please make sure the product is completely dry before each use..

FITTING INSTRUCTIONS

1850CH - ZIPPED HARNESS

EN

With the user seated, place the harness with the zipper closed around the abdomen, then pass the lower straps around the waist to the back of the couch or chair. Pass the straps over the shoulders until you reach the closure, guided by the colored marking on the straps to place them in the correct position. Tighten the straps at the end to adjust the fit. Once in place, use the zipper above to release the user.

1852CH - CLOSED HARNESS

Place the harness around your abdomen, then loop the bottom straps around your waist to the back of the couch or chair. Pass the straps over the shoulders until you reach the closure, guided by the colored marking on the straps to place them in the correct position. Tighten the straps at the end to adjust the fit.

1854CH - COMPLETE PERINEAL HARNESS

With the user seated, place the harness around the abdomen, then loop the lower straps around the waist to the back of the recliner or chair. Pass the straps over the shoulders until you reach the closure, be guided by the color markings on the straps to place them in the correct position. Tighten the straps at the end to adjust the fit. Then pass the perineal strap between the legs, insert it between the slot in the backrest and the seat until it closes if the type of seat allows it, if not, hug the chair from the outside. Close the perineal strap guided by the colored marking, then tighten to the desired fit.

1856CH - BASIC BELT

Place the belt around your abdomen, hug the straps to the backrest and close them. Finally, adjust the desired support by tightening both straps.

1858CH - PERINEAL BELT

Place the belt around your abdomen, hug the straps to the backrest and close them following the color system. Pass the inner strap between the thighs under or through the seat until you reach the closure. Finally, adjust the desired support by tightening the straps.

1860CH - LEG ABDUCTOR

Place the lumbar plate on the user's buttocks with the rattier straps touching the backrest area, then pass the straps around the abdomen, close them and regulate the pressure.

Then wrap around the thighs with the strap on the same side from the inside through the crotch to the outside over the thigh to the sides. Join the rattier straps at the ends behind the chair and adjust the tension using the pin.

1865CH - WRIST HARNESS

Adjust the wristband to the user's forearm using the Velcro closures. Pass the rattier strap through the bed frame, securing it with the buckle, checking that it adjusts correctly. Next, adjust the length of the strap until you obtain the optimal fit for limiting the user's movement.

Keep in mind that the wrist harness has the option of adapting to the armrest of the chair as long as its design allows the strap to completely embrace them.

NOTE: You can add the strap extraction to all references.

1858AC - STRAP EXTENDER

Once the user is sitting in the wheelchair, pass the strap extender around both sides of the wheelchair. Adjust the size so that it fits and provides abdominal support before closing the buckle.

RECOMMENDATIONS

Keep in mind that the material is flammable, so it is important not to expose these products to a situation that could cause it to ignite. If so, quickly remove the product.

In case of allergies, scratches, irritation or swelling, remove the product and consult the prescribing doctor or orthopaedic technician. Do not use on open scars with swelling, redness, or heat build-up.

GUARANTEE

Teyder® guarantees its products, as long as these have not been manipulated from their original state. Product will not be covered under guarantee in case of misuse, not following the recommendations presented on this leaflet. If you detect any abnormally on the product after purchasing it, please inform immediately the establishment where the product was purchased in order it to be changed.

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PRIM, S.A.