



UP
GO!


NEW

Drop foot orthosis
**Don't let anything
stop you**





PRIM UPGO is an innovative, dynamic orthosis designed to provide functional support for the dorsal flexion, controlling medio-lateral deviations of the foot, during the swing phase of gait in cases of foot drop caused by flaccid paralysis.

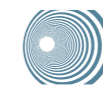

High frequency technology



3 components

1 Supramalleolar band made of flexible and stabilising fabric that adapts by means of a micro-velcro closure with integrated double adjustment. Anatomical silicone padding to release and protect the Achilles tendon.

2 Shoe support with microvelcro fastening which fits both lace-up and Velcro footwear without the need for metal hooks.


3 An innovative strap-and-buckle traction system connects the support to the band and keeps the foot in a physiological position.

► Innovation for the benefit of the patient: comfort and precision

The PRIM UPGO drop foot orthosis has an exclusive strap-and-buckle traction system with magnetic buckles and locking tabs to precisely regulate the dorsal flexion and correct ankle deviation whilst ensuring patient comfort.

INDIVIDUAL ADJUSTMENT OF MEDIUM-LATERAL CORRECTIONS

Its elastic straps allow the degree of dynamic correction to be adjusted according to the needs of each patient, allowing the ankle varus/valgus foot to be slightly corrected, by tightening each of its straps to a greater or lesser extent, or by exerting central traction if no lateral component is to be corrected.

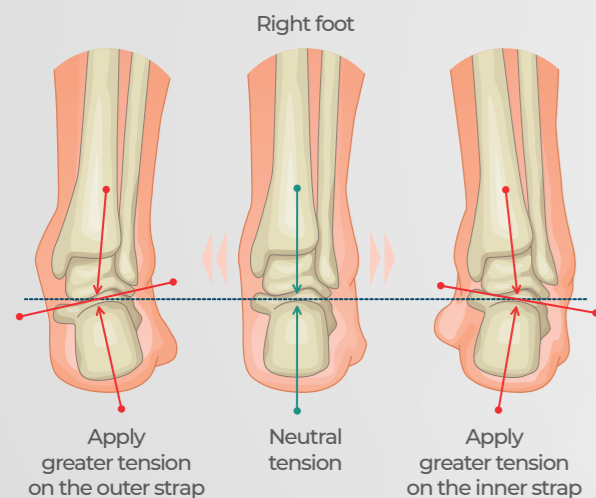

The straps have two areas of elasticity: the first provides constant elasticity that facilitates the positioning of the orthosis, and the second provides a braking effect that helps to control plantar flexion.



EASYCLICK SYSTEM

ONE-CLICK FITTING

PRIM UPGO's unique, patented* magnetic buckles can be adjusted in one click so that the patient can fit and remove the orthosis independently, with just one hand (in cases of hemiplegia and hemiparesis).

*Patent being processed.



MEMORY EFFECT

These buckles incorporate a locking system that keeps the degree of correction performed by the adapter fixed, without the patient having to adjust it.



Unique

VERSATILE

The PRIM UPGO shoe support has been designed so that it can be attached to any type of shoe, either with laces or velcro straps.



CONFIDENCE AND FREEDOM IN EVERY STEP

PRIM UPGO helps dorsiflexion of the foot during the heel strike phase, allowing plantar flexion in the take-off phase, for a harmonious, dynamic and safe gait.



No stumbling



Freedom of movement



Feel every step

▶ For an active day-to-day life

Thanks to the comfort and stability offered by the new PRIM UPGO drop foot orthosis, it can be used in various situations, including walking, standing up, going up and down stairs, etc.



IMMEDIATE RELAXATION

To relax the foot and feel more comfortable during moments of leisure when the patient is sitting, the degree of correction can be eliminated by simply removing the connecting buckles, which can be reconnected with just one click after this period of rest.



With or without shoes?

Lightweight, anatomically shaped plantar band, the function of which is to allow the use of the PRIM UPGO drop foot orthosis without the need for shoes. It can be used with open shoes, flip-flops, sandals or even barefoot, but it is recommended to use it with socks.

Plantar band for PRIM UPGO drop foot orthosis



- 1 Foam padding under the sole of the foot for enhanced comfort.
- 2 Simple and secure velcro adjustment system.
- 3 It is positioned in the central area of the sole of the foot and attaches to the supramalleolar band with the same system as the shoe support.

Sold separately



High frequency
technology



PSL8ABD OS
One size.
Fits left or right foot.
Forefoot circumference: 17-29 cm



PSL890

**A unique model.
3 sizes.**

- S** 13 - 20 cm
- M** 20 - 26 cm
- L** 26 - 32 cm



Bilateral · Bilatéral · Bilaterale

100% LATEX FREE ✓



#WhatMovesUs