| |
|---------------------|
| FR |
| Description: |

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| |
|---------------------|
| ENG |
| Description: |

Ankle orthosis with removable side stabilisers and figure-of-eight strapping. Fits left and right.

Indications - Intended use:
Indicated for the treatment of mild and moderate ankle sprain, ankle tendinitis and injury, bruising of the joint, chronic irritation, post-surgical treatment, osteoarthritis, ankle arthritis and instability.

Contraindications:

- Do not use on open wounds with swelling.
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.

Contraindications:

- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop that sold you the product.

For the orthosis to serve its purpose, it must be correctly adjusted to the patient's morphology. The first fitting and adjustment must be carried out by qualified personnel (doctor, orthopaedic technician, etc.).

Always follow these general instructions of use and any particular guidelines indicated by your general practitioner.

Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.

If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately.

Your doctor might consider the use of topical creams in conjunction with the orthosis.

In areas where pressure is applied, the skin should not be damaged or hypersensitive.

Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.

For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.

To ensure they function correctly, clean the hook-and-loop fastenings regularly, removing any material that may have stuck to them.

Clean the product regularly and maintain proper hygiene.

Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.

Do not let other children play with this device.

Instructions for proper fitting:

It is essential that you select the correct size for the patient.

Insert valves into the side pockets (Fig. 1).

Open the velcro fasteners and position the foot so that the heel is over the opening (Fig. 2).

Align the velcro fasten on the lower section of velcro above the instep (Fig. 3).

Tighten the upper fasteners and attach the sections of velcro (Fig. 4).

Position the strapping in a figure of eight over the ankle, attach the end without tightening (Fig. 5), pull the strap around the edge of the ankle (Fig. 6) and fasten on the opposite side, applying the desired level of tightness (Fig. 7).

Product maintenance - washing instructions:

This product is latex-free. Before washing, fasten all of the velcro fasteners. Wash with a neutral detergent at a maximum of 30°C. Do not dry clean. Do not iron. Do not use bleach.

Do not tumble dry. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Remove the stabilisers before washing, clean them with a damp cloth, then dry.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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