Antiequino textil

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Textile Fußheberorthese

Klumpfodsbandage i tekstil

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Description and intended use:

PSL890 Fabric drop-foot orthosis: Made of high density fabric materials and lined with optimum materials for contact with the skin. Gives the patient extremely comfortable, correct drop-foot support. With a buckle especially designed for easy fitting and removal with just one hand. Can be worn with or without socks.

PSL8ABD plantar band for indoors: Made of extremely soft, cushioned materials, it allows for the drop-foot orthosis to be worn without the need for footwear, designed for indoors and for use with flip-flops. Must be worn with socks.

PSL8ASP full drop-foot support: Spare shoe attachment piece with set of straps and buckles. Designed as piece replacement or for insertion in several shoes to avoid having to remove the original piece.

Indications:

PSL890. Helps raise the forefoot in drop foot during active walking: in the event of flaccid limitation in dorsiflexion of the foot. For the treatment of neurologic lesions presenting with drop foot, such as hemiplegia, consequences of flaccid paralysis, and common peroneal nerve lesions.

Contraindications:

- Do not use on open wounds with swelling
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Only use in low footwear that does not interfere with the main part of the orthosis when in use. Avoid wearing boots and ankle boots.
- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the supplier you purchased it from before use.
- For the orthosis to be effective, patients must use the appropriate size. Choose the next size up if the patient is close to the maximum range of a size and it does not wrap around or barely wraps around the ankle. Choose the next size down if the patient is close to the minimum range of a size and the orthosis turns when it is closed.
- Always follow these general instructions for use and any particular guidelines indicated by your doctor.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately. Discontinue use

until assessed by a specialist.

- Your doctor should consider the use of topical creams in conjunction with the orthosis.
- The skin should not be damaged or hypersensitive in the areas where pressure is applied.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The first fitting and adjustment of these accessories must be carried out by qualified personnel (doctor, orthopaedic technician etc.).
- For the orthosis to function correctly, its components must be regularly checked to ensure they are in good condition. The medical personnel responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- To ensure they function correctly, clean the hook-and-loop fastenings regularly, removing any material that may have stuck to them.
- The product is not flame retardant; therefore, its use is not recommended in environments with excessive heat, fire or radiation.
- Clean the product regularly and maintain adequate personal hygiene.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.
- Do not let children play with this product.

Drop-foot orthosis fitting instructions:

The product must be fitted by your orthopaedic technician or doctor.

To gain the highest degree of therapeutic effectiveness for the different conditions and to prolong the life of the product, it is essential to select the right size for each patient or user (a table of sizes and equivalents in centimetres is enclosed in the packaging).

Check that all the components of the product are present and that it has no kind of anomaly prior to each use. As excessive compression can lead to intolerance, we recommend adjusting the compression to the optimum degree.

Fitting instructions for the foot tensor piece:

1. Loosen the laces on the footwear and the lower micro-hook, placing it between the tongue of the shoe and the laces, making sure it is centred (Fig. 3 and 4).
2. Pull the D ring on the strap buckle out between the laces (Fig. 4)
3. Secure the lower micro-hook, holding onto the laces and inserting the D ring on the strap buckle through the hole (Fig. 5). The laces or Velcro (depending on the type of footwear) should be tightened correctly after fitting the tensor piece.

Fitting instructions for the foot tensor piece in footwear with velcro fasteners (Fig. 19)

1. To place in the distal space of the shoe if the footwear allows as such, open the velcro fasteners and place the buckle on the end of the tongue, placing the piece between the tongue and the sides of the sneaker. (Fig. 20)
2. Close both velcro fasteners, holding onto the tensor piece and making sure the velcro fasteners are tight. (Fig. 21)
3. Secure the tensor piece using its own velcro fastener to prevent movement during use. (Fig. 22)
4. To place in the distal space of the shoe. If this is not possible, open the first velcro fastener only and place the buckle on the edge of the second velcro fastener, placing the piece between the tongue and the sides of the sneaker. (Fig. 23)
5. Close the 2nd velcro fastener, holding onto the tensor piece and making sure it is tight. (Fig. 24)
6. Secure the tensor piece using its own velcro fastener to prevent movement during use. (Fig. 25)

Fitting instructions for the main part of the orthosis:

1. Position the supramalleolar piece on the rear of the ankle, making sure the centre of the silicon cushion is over the Aquiles tendon (Fig. 1).
2. Wrap the ends of the garment around the ankle (Fig. 2).
3. Secure the widest end of the micro-hook, making sure it does not cover the buckle (Fig.6).
4. Pull and secure the narrowest straps (Fig. 7 and 8). If the upper strap coincides with the buckle, secure it at a downwards angle (Fig. 9).
5. Thread the male buckle-slide of the straps to the female buckle-slide of the foot tensor piece (Fig.10). To prevent wear, do not open this buckle while the straps are tight.
6. Then thread the magnetic buckles to the side buckle supports (Fig. 11). The right-hand strap is marked with a blue yarn so that the tension on each side can be personalised without making positioning errors. Open the tabs on the buckles and pull on the ends of the straps to the desired traction (Fig. 12). Lastly, close the tabs to secure the tension (Fig. 13) and the garment will now be in place (Fig. 14).
7. The garment can be removed/loosened using either of the 2 side buckles. It is not necessary to loosen the straps again by opening the magnetic buckle locking system, which means that the patient will always have the tension prescribed by the specialist whenever the garment is worn.

Indoor band fitting instructions:

1. Fasten the band in the middle of the foot with the Velcro closures located on the instep, ensuring that the securing buckle is facing in the correct direction (Fig. 15).
2. Then adjust the front straps and secure them using the Velcro so that the buckle is centred on the instep (Fig. 16). If necessary, in both this step and the next, place the Velcro over the other if the strap is too long

(Fig. 17).

3. Do the same with the rear straps (Fig. 17).

4. The product is in place and ready to thread the main part in the same manner as when the orthosis is secured to normal footwear (Fig. 18).

Product maintenance - washing instructions:

Close all the micro-hooks before washing. Hand wash with warm water and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Note: Any serious incidents involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.