BXCDAV BXCNCIGHT BXCSPORT

ES

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ENG

Description and intended use:

Bruxicalm is a dental discharge splint which main function is to protect the teeth from involuntary friction when under stress and/ or while sleeping.

Indications:

Bruxicalm Day (BXCDAV): Bruxism. Periods of stress in which teeth grinding occurs involuntarily during the day. Bruxicalm Night (BXCNCIGHT): Bruxism. Periods of stress in which teeth grinding occurs involuntarily during the night. Bruxicalm Sport (BXCSPORT): Bruxism. Periods of stress in which teeth grinding occurs during sports activities.

Precautions:

- The splint is recommended for use with adult teeth.
- Do not wash the splint in hot water.
- Avoid storing the splint near heat sources and at temperatures higher than 60 °C.

A dentist or physician should be consulted before beginning to use a dental splint. This applies in particularly to the following cases:

- Severe bruxism.
- Dental or gingival conditions Gum disease.
- Respiratory or other sleep-related health problems.
- Temporomandibular joint pain and sounds.
- Temporomandibular joint disorders, also known as temporomandibular dysfunction, craniomandibular dysfunction (CMD).
- Minors.

The splint should not be used in case of:

- Broken or loose dental repairs, such as fillings, crowns or veneers.
- If, simultaneously, it is used dentures or braces.
- If there are loose teeth.

The device must be revised periodically. If you detect any type of unusual wear, you must replace it immediately.

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the member state where the user and/or patient is located.

Instructions for proper fitting:

The fitting must be made on the upper teeth.

Steps to follow for the correct fitting of the splint:

- Heat water 70 °C.
 - Remove the container from the heat source.
 - Hold the splint by the tab with the product logo and immerse the splint completely in water for 12 seconds (BXCDAV) y 30 seconds (BXCNCIGHT & BXCSPORT) as shown in FIG.1.
 - Slightly widen the walls of the splint FIG.2 in order not to bend them when inserting the splint into the mouth; be careful not to bite into bent walls. Warning: Before inserting the splint into your mouth, shake it to remove any water.
 - Insert the splint into your mouth as shown in FIG.3, close your mouth without clenching your teeth. With closed lips, breathe in and suck in air for one minute to mould it to the shape of your teeth. Press firmly with your fingers simultaneously for a better fit.
 - After one minute, remove the splint from your mouth and place it in cold water for at least 20 minutes FIG.4. The device must hold itself in place, if not repeat the steps from 1.
 - Finally, cut the tab with the product logo flush with the splint as shown in FIG.5.2
 - In case the splint is too long, cut the ends with the help of the cut marks FIG.5.1. If any creases or kinks in the material appear after fitting, remove those using scissors.
- Product maintenance - washing instructions:**
After each use, wash the splint with a toothbrush under cold water. The device has a lifespan of 12 months.

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PRIM, S.A.