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We take care of your feet for you

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ComforSil[®]

CC210

ComforSil

(ES) CC210 COMFORSIL PLANTILLAS FASCITIS PLANTAR

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(ENG) CC210 COMFORSIL PLANTAR FASCIITIS INSOLES

1. Description and intended use

These insoles are specially designed to provide relief from the pain associated with plantar fasciitis. Their unique design combines support for the arch of the foot, cushioning for the heel and a breathable upper layer to provide comfort throughout the day. These insoles are intended for use by people who are suffering from plantar fasciitis and want to lessen the pain and discomfort they experience while walking, standing, and performing other everyday activities.

2. Indications

These insoles are indicated for people who are experiencing pain in the heels and/or arches of their feet due to plantar fasciitis.

These insoles can be used with a variety of footwear, including sports shoes, dress shoes, boots and any other type of footwear, provided they do not have heels, cover the instep of the foot, and have sufficient space to accommodate the insoles.

3. Contraindications

- These insoles are not recommended for use if your feet are significantly deformed.
- These insoles should not be used if you have open wounds, cuts, blisters or any type of skin irritation on your feet.
- If you suffer from blood circulation issues affecting your feet, consult a doctor before using these insoles.
- If your symptoms get worse or new problems develop after you have begun to use these insoles, stop using them and consult a doctor or podiatrist.
- If you are suffering from a loss of sensitivity in your feet due to conditions such as diabetic neuropathy, use these insoles with caution and under the supervision of a doctor or podiatrist, in order to prevent undue discomfort or pressure.
- If you are in any doubt regarding the suitability of these insoles, consult a doctor or podiatrist.

ComforSil

4. Warnings

- Read these instructions carefully before using the insoles. If you have any queries, consult your podiatrist or the establishment where you purchased the product.
- These insoles are only intended for sufferers of plantar fasciitis. Do not use them to treat other conditions without consulting a doctor.
- Although the insoles are not single-use, they should only be worn by a single user. Not to be reused by others.
- Start with an adaptation period by using the insoles for a few hours, and then gradually increase the amount of time you use them and the intensity of the activities you carry out while using them.
- Select the correct size and, if necessary, carefully trim the insoles along the marked lines in order to ensure they fit comfortably in your shoes.
- Use these insoles in shoes that are roomy enough to accommodate them, in order to prevent the shoes from becoming too tight.
- Clean the product regularly and maintain adequate personal hygiene.
- Keep out of reach of children. Small components, such as the silicone cushion, may constitute a choking hazard.
- Dispose of the product in accordance with the local environmental regulations in force.

5. Instructions for use

1. Select the correct size.
2. If necessary, trim the insoles following the raised marks on the underside of the EVA rubber. Trim the insoles little by little, and try placing them inside the shoes until they fit comfortably without folding or wrinkling. (Fig. 1)
3. Remove the existing insoles. If the original insoles are stuck on and cannot be removed easily, try to place the new insoles on top of them. Make sure that the shoes are not too tight.
4. Slide the insole inside the shoe, making sure that the heel of the insole is correctly positioned at the rear of the shoe. The upper fabric side of the insole must be facing upwards, so that it is in contact with the foot. (Fig. 2)
5. Make sure you feel adequate support beneath the arch of the foot, and that the silicone cushion is correctly aligned with the heel and provides cushioning. The insole should not feel uncomfortable or move about inside the shoe.
6. These insoles can be used with various types of footwear, such as sports shoes, dress shoes and boots. Always make sure there is enough space inside the shoe to accommodate the insole without the shoe being too tight.

6. Product maintenance and washing instructions

Hand wash using a damp cloth and mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose the insoles to direct heat sources such as stoves, heaters, radiators and direct sunlight.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.