




Dental guard

for operating theatres

One Size 
Single use only

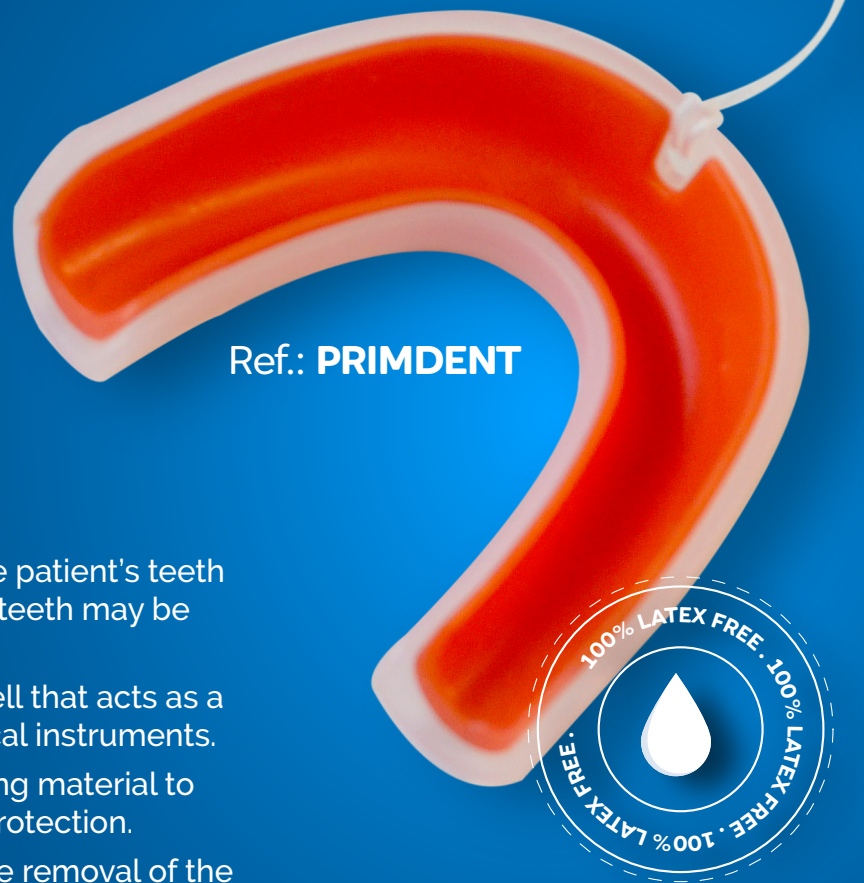

Ref.: PRIMDENT



Dental guard

- Dental guard designed to protect the patient's teeth during medical procedures in which teeth may be damaged.
- Manufactured with a tough outer shell that acts as a barrier between the teeth and medical instruments.
- With an inner layer of shock-absorbing material to cushion impacts and provide extra protection.
- Includes a safety cord to facilitate the removal of the guard when necessary.



Indications

- Endoscopic surgery
- Procedures requiring the use of the laparoscopic blade.
- Dental protection.

Dental guard

for operating theatres

Ref.: **PRIMDENT**

Dental guard



The disposable, **single-use** dental guard is an appropriate medical device for **protecting a patient's teeth during the insertion of a laryngoscope blade or in endoscopic surgery** using an endotracheal tube, in which the patient's teeth could be damaged.

The use of dental guards during intubation has been advocated as a **preventive measure to reduce the risk of dental injury**.

Adaptable to almost any patient.

Each set comprises:

- **Highly malleable extra-soft silicone filling** that adapts to any teeth shape in order to absorb the pressure on the gums. Atraumatic. (1)
- **Semi-rigid outer shell** with sufficient flexibility to fit all teeth. (2)
- **Safety cord** for extraction. (3)

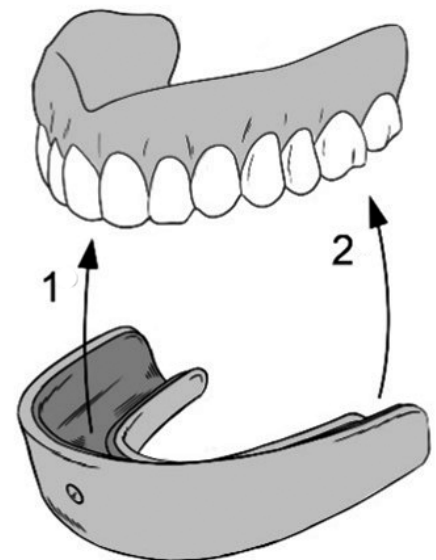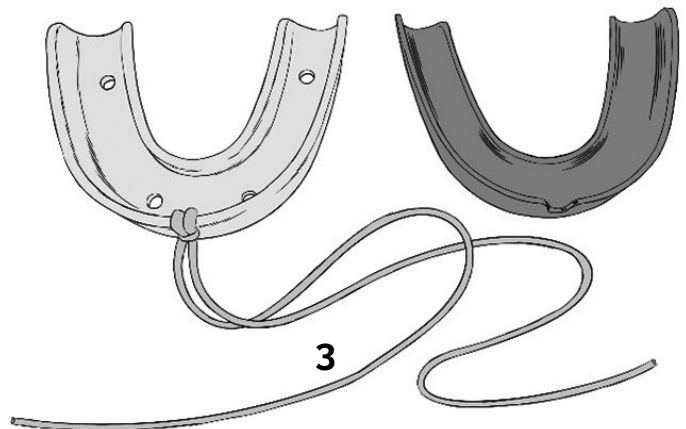

Contraindications

- Non-sterile product. Do not use when sterile conditions are required.
- Do not use in children or patients with very small dental arch sizes.
- Not recommended for use in patients with dentures.
- Do not use to treat dental conditions such as bruxism.
- Do not use in patients with advanced dental mobility.

PRIMDENT

Code	Pack
844394	1 und
842648	40 und