

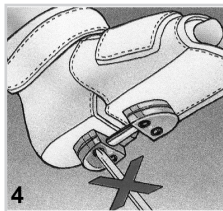
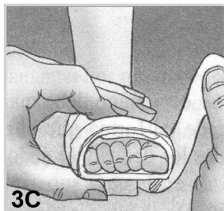
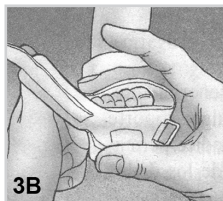
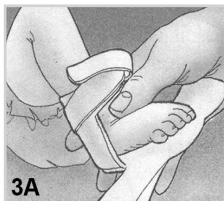
BébaX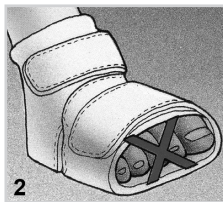

Instrucciones de adaptación · تعليمات المستخدم · Pokyny k nastavení · Anleitung ·
Tilpasning instruktion · Οδηγίες εφαρμογής · Fitting instructions · Sovitusohjeet ·
Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások ·
Istruzioni per l'adattamento · Montage-instructies · Tilpassingsinstruksjon ·
Instrukcja obsługi · Instruções de adaptação · Instructiuni de reglare ·
Инструкции по адаптации · Návod na použitie · Utprovninginstruktion

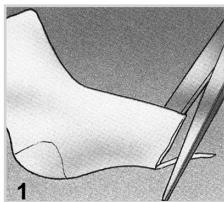5

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The Bebax orthopaedic bootie has been prescribed to your newborn for the treatment of incorrect positioning of the foot.

Bebax bootie is a brace made in leather with two internal structures in injected plastic, padded and joined together by a multidirectional joint.

Bebax allows progressive correction of congenital forefoot deformities in all three space planes.

Indications:

Metatarsus varus or metatarsus adductus. Talus Valgus feet. Talus feet or Valgus feet. Residual forefoot varus after treatment of clubfoot. Pes supinatus (varus feet).

How to use BEBAX:

- The correction obtained by Bebax is comparable to the correction obtained from plaster casts. However, Bebax treatment allows removal of the device for examination, bathing and exercise. Also, with Bebax, the problems of breakdown and foot stiffness are less likely.
- The fitting and adjustment of the Bebax bootie should be realized by the doctor or orthopaedic technician.
- It is important not to over-correct. If corrected slowly and gradually, you will get faster results.
- Do not over-tighten the screws to avoid damaging the joint.
- During the first month of treatment, your doctor will ask you to make regular office visits to control the progress of the treatment and to make necessary adjustments.
- In order to prevent relapses, it is advisable to keep the use of Bebax for about two weeks after the pathology reduced.

How to fit BEBAX:

- The Bebax bootie should be used with a thin cotton sock, which you have cut in the toe area, so you can check the colour of the baby's toes (**figure 1**). Check the circulation of the toes each time you place the bootie (**figure 2**).
 - Place the baby's heel at the back of the Bebax bootie. While holding the heel in place, hold the ankle of the child as shown on (**figure 3A**).
 - Then place the baby's toes into the front part of the bootie. Be sure that the heel remains snug (**figure 3B**).
 - Pass the toe strap through the buckle and fasten while holding the toes (**figure 3C**).
 - Do not change the correcting position realised by your doctor (**figure 4**).
 - A loose fitting Bebax may slow down the correction. Do not, therefore, hesitate to make the Bebax fit fairly snug.
- * For the larger size (sizes 10,5 - 11,5 - 12,5 and 13,5), pass the strap through the slit and band provided for it (**figure 5**). While holding the ankle with your thumb, tighten and fix this strap firmly, then fasten the ankle strap.

Advices:

- The duration of treatment with the Bebax bootee depends on the nature and the importance of the deformity. The doctor is the only person qualified to decide the duration of the treatment. He will increase or reduce the degree of correction on Bebax and decide the daily time of use.
- To achieve a therapeutic effect, you should respect the daily time of use prescribed by your doctor. Once the bootee placed, do not change the correcting position. If the screws come loose, consult your doctor. Although Bebax is well tolerated by the children, if you notice reddening on the skin, it is advisable to inform the doctor who is treating your child.
- The colour of the toes may be rosy or slightly blue. If the toes are very blue or white, wait a few minutes to see if the toes return to their normal colour. If not, loosen the straps slightly (**figure 4**).
- You can remove Bebax during bathing, placing it back again immediately after. (consult your doctor).
- It is advisable to softly massage the child's foot every time Bebax is removed for any reason. (consult your doctor).
- If the child is capable of walking, Bebax must only be used during the night or during its periods of rest. Do not let the child walk or stand with it as the joint can be broken.
- If you notice that the toes overpass Bebax, you must use a larger size. It is important that Bebax is well fitted to the foot size of the baby so that the support points of correction correspond as perfectly as possible to the anatomical reference points. (consult your doctor).
- Due to the anatomical differences of the foot, BEBAX® orthosis must not be used for more than one treatment. Avoid the use in other patients.
- **Only small sizes of Bebax (from 7,5 to 10) can be mounted on Clubax.**

Maintenance:

- As BEBAX is made in leather, it should be cleaned with a wet cloth.
- The BEBAX bootee is not suitable for autoclave.

Sizes:

- The size is measured by the child's foot length from heel to tip of big toe.
- There are three Bebax bootie models: small, large and extra large, which correspond to the following sizes:
- Small sizes: 7,5 – 8 – 8,5 – 9 – 9,5 – 10 cm (left or right).
- Large sizes: 10,5 – 11,5 – 12,5 – 13,5 cm (left or right).
- Extra large: sizes: 14,5 – 15,5 – 16,5 cm (left or right).
- Bebax Bootee sold by unit, right or left.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A